

How to Use the CartWashCheck™

1



The CartWashCheck™ is the first test designed to challenge the mechanical efficiency of the cart washer. These convenient tests can be quickly and easily adhered to any metal surface with the peel off adhesive that leaves no residue.

Unexposed:

*Transportation and Storage
Conditions:
Keep Below 100°F &
Avoid Exposure to Moisture.*



2



Place these tests on surfaces which are furthest from the washer jets or in areas where you suspect coverage is most difficult. The special hydrophilic ink will change color (from black to white) only if moistened by water. This demonstrates that water is reaching the area tested and indicates proper mechanical action by the washer.

White Means Water Reached the Test

3



To ensure that your washer is achieving your target temperature setting, the CartWashCheck also includes an irreversible thermometer. The thermochromatic (heat sensitive) indicator above the hydrophilic ink will register temperature levels of 120°F, 150°F, and 180°F, documenting the temperature reached during the cycle.

*The Temperature Gauge Means the Surface
Reached at Least the Corresponding Temp.*



healthmark
INDUSTRIES CO.
health care products
800-521-6221
www.hmark.com

Code No.	Description	Qty
CW-101	CartWashCheck	50