


TOSI[®]-FlexiCheck Instructions for use

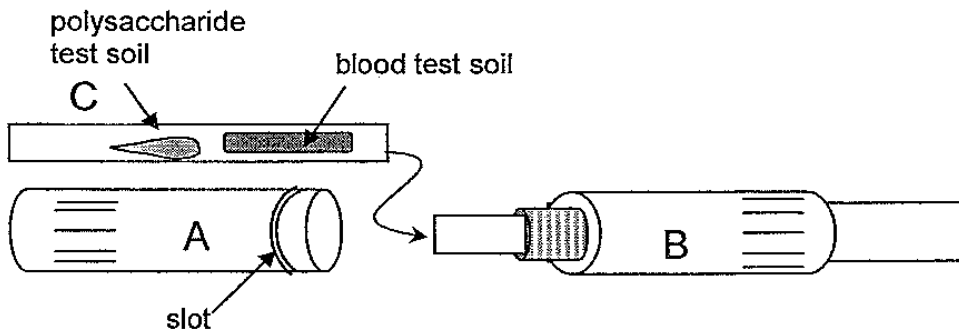

1. Unscrew FlexiCheck-device: detach part A (marked with slot) from part B (see picture).
2. Open protective pouch of TOSI[®]-FlexiCheck and insert the test object (part C) in part B as shown in picture. Do not touch the area covered with test soil.
3. Close FlexiCheck-device again.
4. Connect FlexiCheck-device with one of the channel irrigation system of the washer (luer-lock) and start the cycle according to your manufacturer manual instructions.
5. Open the device after the reprocessing cycle: disconnect part A from part B to remove the TOSI[®]-FlexiCheck without touching the test soil area!
6. Visual inspect the result of the TOSI[®]-FlexiCheck.

STORAGE:

CONTENT: 25 tests

Store in protective pouch at 2°C - 25°C

Do not open TOSI[®]-FlexiCheck protective pouch before use

Shelf life: see imprint

Marketing & Sales:

Healthmark Industries
33671 Doreka Drive
Fraser MI. 48080 USA

Manufacturer:

PEREG GMBH
Porschestr. 12
D-84478 Waldkraiburg