








TOSI® – FlexiCheck Troubleshooting Guide

	<p>Positive result: Both test-soils completely removed.</p>
	<p>Result: Polysaccharid-test soil completely removed but visible residue of fibrin left. Indication for: Protein dissolving parameters not optimal Optimisation: Check and/or correct: Cleaning time / temperature / detergent efficiency / dosing</p>
	<p>Result: Blood-test soil completely removed but visible residue of Polysaccharide left. Indication for: Polysaccharide dissolving parameters not optimal Optimisation: Check and/or correct: Water quality / detergent efficiency and dosing / Cleaning time and temperature</p>
	<p>Result: Polysaccharide-test soil completely removed but red protein residue left. Indication for: Protein denaturing effects (heat or disinfecting agents) Optimisation: Check for high temperature or disinfecting agents during the wash cycle. Run cold pre-rinse if possible.</p>
	<p>Result: Both spots of test soil completely left. Indication for: Missing flow / No cleaning efficiency. Optimisation: Check connection of FlexiCheck , machine and cleaning program.</p>