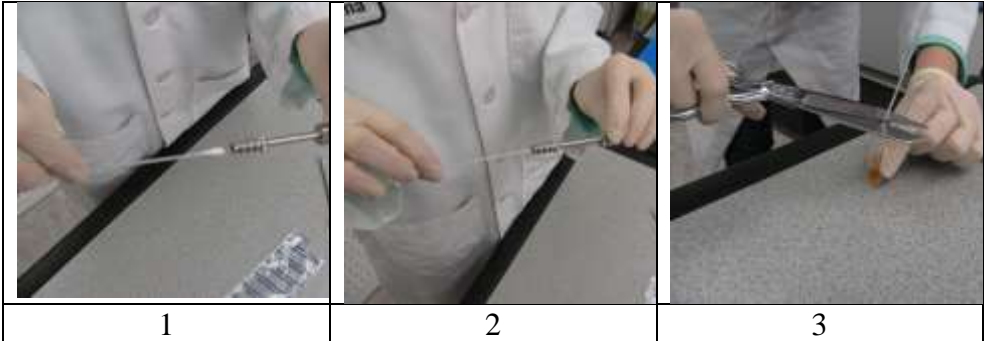


Instructions for Use: Clean Test Swabs

| | |
|--|--|
| Brand Name of Product | Clean Test Swabs |
| Generic Name of Product | Swabs for extraction of residuals for testing |
| Product Code Number(s) | SWTD-110-12, SWTD-200-12, SWTD-270-12, SWTD-350-12, SWTD-420-12, SWTD-470-12 |
| Purpose of Product | For use in combination with ProChek®-II or HemoCheck™ to verify cleanliness. |
| Range of Applications for Product | Detection of blood or protein residues on surfaces. These swabs are for use with conjunction with the ProChek®-II or HemoCheck™ to verify cleanliness. |
| Key Specifications of Product | |

| Shipping & Storage | |
|---|--|
| Shipping Conditions & Requirements | |
| Storage Conditions | |
| Packaging Conditions | |
| Shelf Life | |

| Instructions for Using Product | |
|--|--|
| Description of Use(s) | These swabs are for use with conjunction with the ProChek®-II or HemoCheck™ to verify cleanliness. |
| Preparation for Use | Select the swab based upon the diameter and length of the channel to be tested. |
| Diagrams (drawings, pictures) |  <div style="display: flex; justify-content: space-around; margin-top: 5px;"> 1 2 3 </div> |
| Steps for Use of Product | <ol style="list-style-type: none"> 1. Wet surfaces are swabbed with the dry swab. Dry surfaces are swabbed by moistening the swab with a drop of clean water (Do not use chlorinated water!). Swab the target area. 2. Be sure to extend the swab beyond the end of the item. 3. Cut the stem with supplied scissor into the reagent vial (see IFU for the relevant reagent test). 4. Depending on the reagent test being used, follow the ProChek®-II or HemoCheck™ IFU for testing instructions. |
| Interpretation of Results | |
| Contraindications of Test Results | |
| Documentation | |
| Special Warnings and Cautions | |
| Disposal | Since it is possible that contaminants are present, it is recommended to dispose of the Clean Test Swabs in a biohazard container. |

| Reprocessing Instructions | |
|--|--|
| Point of Use | |
| Preparation for Decontamination | |
| Disassembly Instructions | |
| Cleaning – Manual | |

| | |
|---|--|
| Cleaning – Automated | |
| Disinfection | |
| Drying | |
| Maintenance, Inspection, and Testing | |
| Reassembly Instructions | |
| Packaging | |
| Sterilization | |
| Storage | |
| Additional Information | |

| | |
|--|---|
| Related Healthmark Products | ProChek® -II or HemoCheck™ |
| Other Product Support Documents | HemoCheck™/ProChek® Log Sheet, HemoCheck™/ProChek® User Guide, PB-Reagent Tests, QA Tray Policy |
| Reference Documents | Improving the Quality of Surgical Trays, Kovach; Loaner Instrumentation – Processing the Unknown, AORN Journal; Study on HemoCheck and EndoCheck, Pfiefer |
| Customer Service Contact | Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com |

2019-08-09 Ralph J Basile

Rev. B