
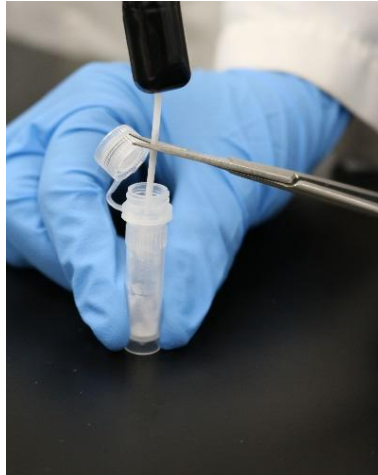


## Instructions for Use: HydroCheck™ Moisture Detection Test


<b>Brand Name of Product</b>	HydroCheck™
<b>Generic Name of Product</b>	HydroCheck™
<b>Product Code Number(s)</b>	HYD-200, HYD-270, HYD-350, HYD-470
<b>Intended Use</b>	To test for residual moisture.
<b>Range of Applications for Product</b>	Detection of residual moisture.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>HydroCheck™ test</li> <li>Swab: Available sizes swab sizes: 1.7, 2.8, 3.8, and 5.0 mm</li> <li>Measuring range - the test kit can detect <math>\geq 0.05 \mu\text{L}</math> of water.</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	Store in closed box at room temperature (15 °C-25 °C) in a dry place.
<b>Packaging Contents</b>	N/A
<b>Shelf Life</b>	N/A


<b>Instructions for Using Product</b>	
<b>Description of Use(s)</b>	To test for residual moisture.
<b>Preparation for Use</b>	<ul style="list-style-type: none"> <li>Put gloves on before using the HydroCheck™ test.</li> <li>Select the correct swab size for the channel to be tested.</li> </ul>
<b>Diagrams (drawings, pictures)</b>	 <p style="text-align: center;"><b>Figure 1</b></p>



**Figure 2**

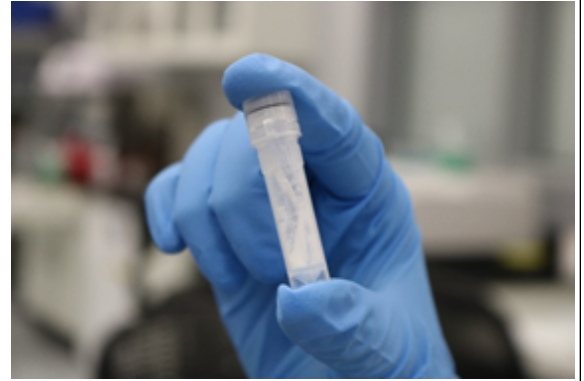

**Figure 3**

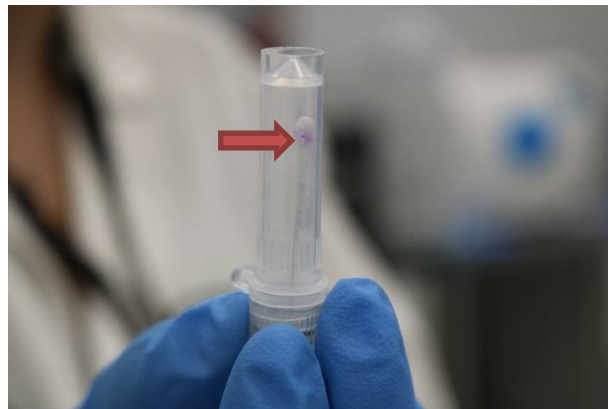

**Figure 4**

<p><b>Steps for Use of Product</b></p>	<ul style="list-style-type: none"> <li>● Insert the swab into the channel to be tested and feed it all the way through the channel until it exits the distal tip. <b>Fig. 1</b></li> <li>● Immediately after the swab exits the distal tip, cut the swab approximately 2 cm behind the tip with scissors directly into the test vial. <b>Fig. 2</b></li> <li>● Close and gently shake at least 3 times. <b>Fig. 3</b></li> <li>● Reprocess the sampled item after testing with the HydroCheck™.</li> </ul>
<p><b>Interpretation of Test Results</b></p>	<ul style="list-style-type: none"> <li>● A color change to purple on the swab indicates residual moisture at or above the level of detection.</li> <li>● A small amount of residual moisture on the tip of the swab will be indicated by a small purple spot. <b>Fig. 4</b> Large amounts of moisture can turn the whole test purple.</li> </ul>
<p><b>Contraindications of Test Results</b></p>	<ul style="list-style-type: none"> <li>● Gloves must be worn throughout the test procedure to avoid contamination of the test.</li> <li>● Contact with any source of moisture (e.g. a fingerprint) can give a false positive result.</li> <li>● If the swab is not tested immediately, it may dry out and give a false negative result.</li> </ul>
<p><b>Documentation</b></p>	<p>N/A</p>
<p><b>Special Warnings and Cautions</b></p>	<p>Do not open HydroCheck™ test before use.</p>
<p><b>Disposal</b></p>	<p>Single-use</p>

**Reprocessing Instructions**

<b>Point of Use</b>	N/A
<b>Preparation for Decontamination</b>	N/A
<b>Disassembly Instructions</b>	N/A
<b>Cleaning – Manual</b>	N/A
<b>Cleaning – Automated</b>	N/A
<b>Disinfection</b>	N/A
<b>Drying</b>	N/A
<b>Maintenance, Inspection, and Testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	N/A
<b>Sterilization</b>	N/A
<b>Storage</b>	N/A
<b>Additional Information</b>	Single-use. One test per channel.
<b>Related Healthmark Products</b>	N/A
<b>Other Product Support Documents</b>	Cleaning Verification Brochure, Cleaning Verification Price List
<b>Reference Documents</b>	N/A
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> hmark.com