


# TOSI<sup>®</sup>-LumCheck

## Instructions for use

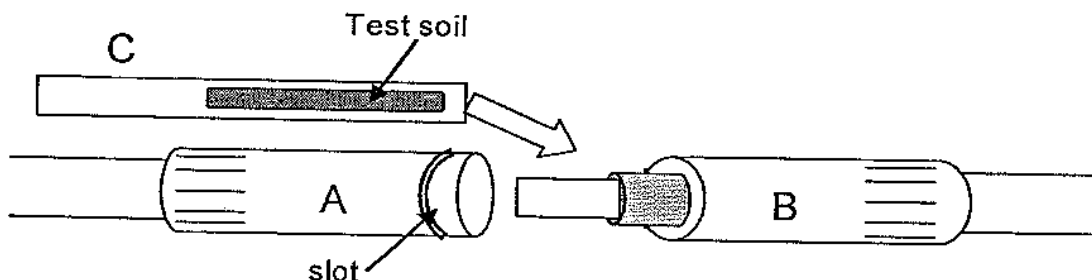

1. Unscrew LumCheck-device: detach part A (marked with slot) from part B (see picture).
2. Open protective pouch of TOSI<sup>®</sup>-LumCheck and insert the test object (part C) in part B as shown in picture. Do not touch the area covered with test soil.
3. Close LumCheck-device again.
4. Connect LumCheck-device with the rinse system of the washer (for example luer-lock) or reprocess it in the same way as done with cannulated instruments.
5. Open the device after the reprocessing cycle: disconnect part A from part B to remove the TOSI<sup>®</sup>-LumCheck without touching the test soil area!
6. For visual evaluation of the result, use the TOSI<sup>®</sup> evaluation table.

**Storage:** Store in protective pouch at 2°C-25°C  
Do not open protective pouch before use  
Keep away from heat, light and chemicals

**Content:** 25 tests

**Shelf life:** See imprint

**Distributor:**  
Healthmark Industries  
33671 Doreka Drive  
Fraser MI. 48080 USA

**Manufacturer:**  
PEREG GMBH  
Porschestr. 12  
D-84478 Waldkraiburg  
Germany