

Instructions for Use: Inspection Results for CartWashCheck™

Brand Name of Product	Inspection Results for CartWashCheck™
Generic Name of Product	Inspection Results for CartWashCheck™
Product Code Number(s)	
Intended Use	To log the information with the inspection results after the CartWashCheck™ was completed.
Range of Applications for Product	Record the results of the inspection after the CartWashCheck™ was completed.
Key Specifications of Product	<ul style="list-style-type: none"> ● ProFormance™ QA 2.2 ● Facility Admin/User ● Washer Test Logs ● CartWashCheck™ ● With Inspection Results

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	
Shelf Life	

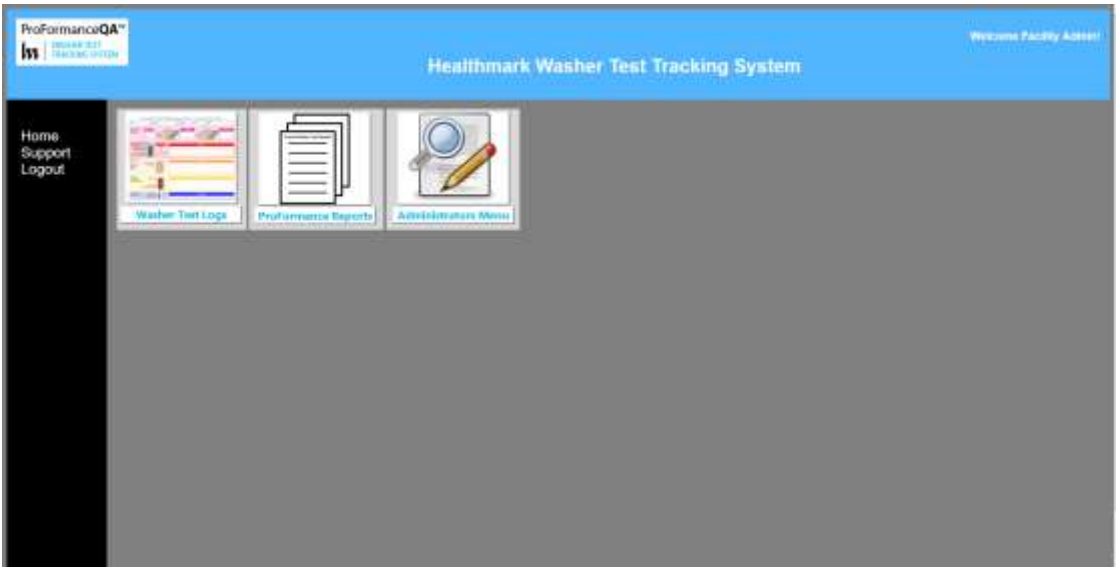
Instructions for Using Product	
Description of Use(s)	Record the results of the inspection after the CartWashCheck™ was completed.
Preparation for Use	Log into the application Facility Admin. in ProFormance™QA 2.2 to access the Washer Test Logs to record the inspection results for the CartWashCheck™.
Diagrams (drawings, pictures)	 <p style="text-align: center;">Figure 1</p>



Figure 2

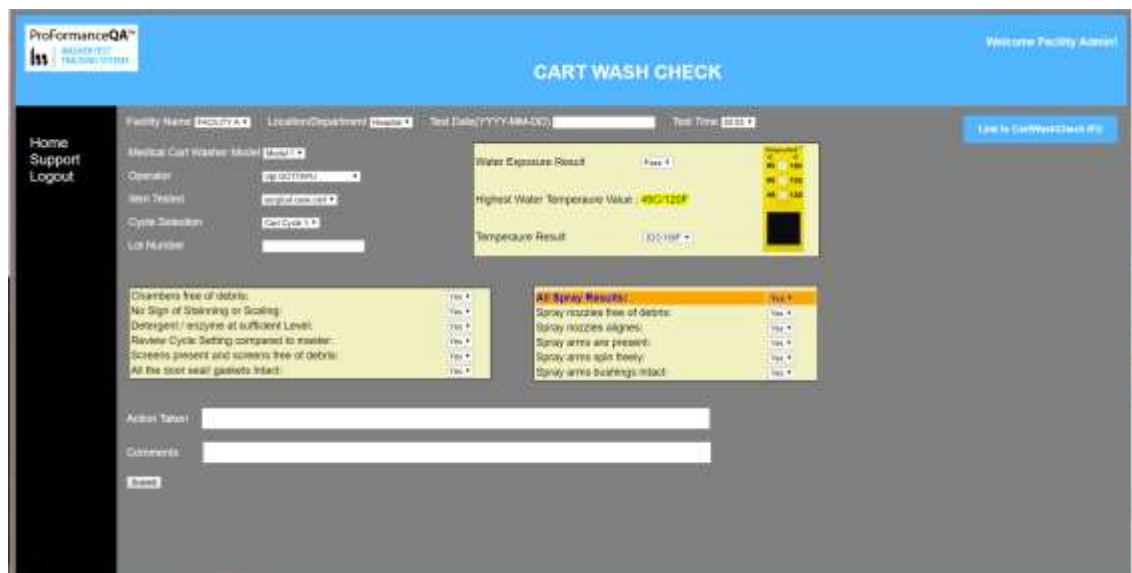


Figure 3

Steps for Use of Product

1. Click on Washer Test Logs. **Fig. 1**
2. Go to CartWashCheck™. **Fig. 2**
3. Then click “WITH INSPECTION RESULTS”
4. The CartWashCheck™ inspection page pops up. **Fig. 3**
5. Here the Operator or Data Entry User will choose from the drop-down list from each section to record results of the inspection.
6. When recording the results of ‘Water Exposure Result’ for the TempaChek™ choose ‘Pass’ or ‘Fail’ from the drop-down list. For ‘Temperature Result’ choose the temperature that the TempaChek™ did reach from the drop-down list.
7. To the top far right, there is a link available for the CartWashCheck™ IFU to reference if necessary. Click on the link and the IFU will come up on a separate page.
8. To close out of the IFU, click on the ‘x’ in the white tab CartWashCheckIFU.pdf to exit. This will take you back to the CartWashCheck™ screen. **Fig.4**



Figure 4

9. Next, inspect the cart wash chamber and record the results.
10. Click on the drop-down list to select 'Pass' or "Fail" for each of these after inspecting the chamber:
 - Chamber free of debris:
 - No sign of staining or scaling:
 - Detergent / enzyme at sufficient level:
 - Review cycle setting compared to master:
 - Screens present and screens free of debris:
 - All the door seal / gaskets intact:
11. Then inspect the spray nozzles and spray arms and record the results.
12. Click on the drop-down list to select 'Yes' or 'No' for each of these after inspecting the spray nozzles and spray arms:
 - Spray nozzles free of debris:
 - Spray nozzles aligns:
 - Spray arms are present:
 - Spray arms spin freely:
 - Spray arms bushings intact:
13. At the bottom of the page there is a section for 'Action Taken', here you can type in any corrective action to be taken.
14. Next in the 'Comments' section, type in any additional information that is necessary.
15. When the page is completed, click on 'Submit' to save results.

Interpretation of Test Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	

Reassembly Instructions	
Packaging	
Sterilization	
Storage	
Additional Information	
Related Healthmark Products	
Other Product Support Documents	ProFormance™QA 2.2
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc. 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2018-10-26 Suzanne Latta