




Figure 4

	 <p>Figure 4</p>
Steps for Use of Product	<ol style="list-style-type: none"> 1. Tear the clear seal open first. Fig. 1, 1A 2. Before use, wipe the neutralizer bottle and the lid with an alcohol wipe. Fig. 2-4 3. Then open the lid of the bottle. 4. Add the D/E Broth to the extracted sample. <ul style="list-style-type: none"> • Do not allow the test object to contact the neutralizer solution in the sample container. • Accidental immersion of any part of the test object into the neutralizer solution necessitates complete reprocessing. 5. Transport the sample to the testing lab.
Interpretation of Test Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	
Storage	
Additional Information	
Related Healthmark Products	
Other Product Support Documents	ProFormance™ Brochure, ProFormance™ Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com