



healthmark
INDUSTRIES CO.
health care products

800-521-6224
www.hmark.com

Instructions for Use: SonoCheck™ Hook

Brand Name of Product	SonoCheck™ Hook
Generic Name of Product	SonoCheck™ Hook
Product Code Number(s)	TI108-HK
Intended Use	The hook securely holds down the SonoCheck™ vial to the basket during the ultrasonic cycle.
Range of Applications for Product	Hold the SonoCheck™ vial in the tray.
Key Specifications of Product	<ul style="list-style-type: none"> • Hook shaped design • Stainless steel • Reuseable hook

Shipping & Storage

Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	
Shelf Life	

Instructions for Using Product



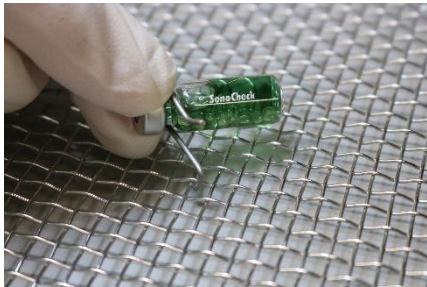

Description of Use(s)	To secure the SonoCheck™ vial in place during testing.	
Preparation for Use		
Diagrams (drawings, pictures)	 <p>Figure 1</p>	 <p>Figure 1</p>
	 <p>Figure 2 (Wire Basket)</p>	 <p>Figure 2 (Wire Basket)</p>



Figure 2 (Perforated Basket)



Figure 2 (Perforated Basket)



Figure 3



Figure 4



Figure 4



Figure 4

Steps for Use of Product

Before the SonoCheck™ test:

1. Place the hook around the neck of the SonoCheck™ capsule.
2. Press the hook down so that the groove of the hook fits securely around the neck of the capsule. **Fig. 1**
3. Insert the L shaped hook on the left side into the hole of the instrument basket to hold down the capsule. The hook can be attached to a wire basket and perforated basket. **Fig. 2**
4. Place the SonoCheck™ in the correct location with the tray/basket according to SonoCheck™ IFU placement based on the size of the tank.

After the SonoCheck™ test is complete:

5. Hold the glass capsule with three fingers of one hand and the index finger and thumb of the other hand on the hook and lid of capsule. **Fig. 3**
6. Place the index finger on the L shaped hook, then place the thumb on the lid of the capsule for support, push the hook back off the neck of the capsule and remove the hook with both fingers. **Fig. 4**

Contraindications of Test Results

Documentation

Special Warnings and Cautions

Disposal

Reprocessing Instructions

Point of Use

Preparation for Decontamination

Disassembly Instructions

Cleaning – Manual

Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	
Storage	Dry after use and store in dry place.
Additional Information	
Related Healthmark Products	
Other Product Support Documents	ProFormance™ Brochure, ProFormance™ Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc. 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2018-04-06 Suzanne Latta