



**healthmark**  
INDUSTRIES CO.  
health care products

800-521-6224  
www.hmark.com



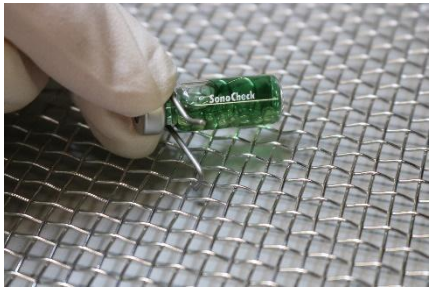

### Instructions for Use: SonoCheck™ Hook

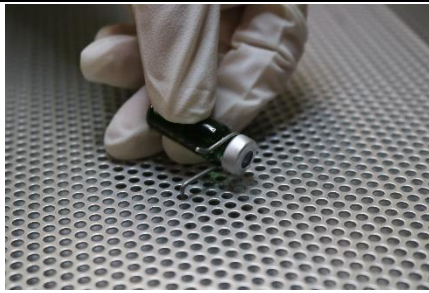
<b>Brand Name of Product</b>	SonoCheck™ Hook
<b>Generic Name of Product</b>	SonoCheck™ Hook
<b>Product Code Number(s)</b>	TI108-HK
<b>Intended Use</b>	The hook securely holds down the SonoCheck™ vial to the basket during the ultrasonic cycle.
<b>Range of Applications for Product</b>	Hold the SonoCheck™ vial in the tray.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"><li>• Hook shaped design</li><li>• Stainless steel</li><li>• Reuseable hook</li></ul>

### Shipping & Storage

<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Conditions</b>	
<b>Shelf Life</b>	

### Instructions for Using Product

<b>Description of Use(s)</b>	To secure the SonoCheck™ vial in place during testing.
<b>Preparation for Use</b>	
<b>Diagrams (drawings, pictures)</b>	 <p><b>Figure 1</b></p>  <p><b>Figure 1</b></p>  <p><b>Figure 2 (Wire Basket)</b></p>  <p><b>Figure 2 (Wire Basket)</b></p>



**Figure 2 (Perforated Basket)**



**Figure 2 (Perforated Basket)**



**Figure 3**



**Figure 4**



**Figure 4**



**Figure 4**

**Steps for Use of Product**

**Before the SonoCheck™ test:**

1. Place the hook around the neck of the SonoCheck™ capsule.
2. Press the hook down so that the groove of the hook fits securely around the neck of the capsule. **Fig. 1**
3. Insert the L shaped hook on the left side into the hole of the instrument basket to hold down the capsule. The hook can be attached to a wire basket and perforated basket. **Fig. 2**
4. Place the SonoCheck™ in the correct location with the tray/basket according to SonoCheck™ IFU placement based on the size of the tank.

**After the SonoCheck™ test is complete:**

5. Hold the glass capsule with three fingers of one hand and the index finger and thumb of the other hand on the hook and lid of capsule. **Fig. 3**
6. Place the index finger on the L shaped hook, then place the thumb on the lid of the capsule for support, push the hook back off the neck of the capsule and remove the hook with both fingers. **Fig. 4**

**Contraindications of Test Results**

**Documentation**

**Special Warnings and Cautions**

**Disposal**

**Reprocessing Instructions**

**Point of Use**

**Preparation for Decontamination**

**Disassembly Instructions**

**Cleaning – Manual**

<b>Cleaning – Automated</b>	
<b>Disinfection</b>	
<b>Drying</b>	
<b>Maintenance, Inspection, and Testing</b>	
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	
<b>Storage</b>	Dry after use and store in dry place.
<b>Additional Information</b>	
<b>Related Healthmark Products</b>	
<b>Other Product Support Documents</b>	ProFormance™ Brochure, ProFormance™ Price List
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 33671 Doreka Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>

2018-04-06 Suzanne Latta