

## Instructions for Use: Mobile Phone

<b>Brand Name of Product</b>	Mobile Phone
<b>Generic Name of Product</b>	Mobile Phone
<b>Product Code Number(s)</b>	PHN-001
<b>Intended Use</b>	<ul style="list-style-type: none"> <li>To help enhance the image that is being visually inspected with the Fiberscope.</li> <li>To log data and track temperature and humidity on the mobile phone for the Temp-DL-TH.</li> </ul>
<b>Range of Applications for Product</b>	
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>Samsung Galaxy S5 SM-G900V 16GB</li> <li>HSDPA 42.2 Mbps, HSUPA 5.76 Mbps</li> <li>129.4 mm (5.1") Full HD Super AMOLED</li> <li>1.9 GHz Quad Core +</li> <li>1.3 GHz Quad Core</li> <li>16 MP AF Camera +, 2 MP Front</li> <li>Water &amp; Dust Resistant</li> <li>GPS</li> <li>2,800mAh Battery</li> <li>Charger</li> <li>Headset</li> <li>Screen Protector</li> </ul>

Shipping & Storage	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Contents</b>	
<b>Shelf Life</b>	

Instructions for Using Product	
<b>Description of Use(s)</b>	To help enhance the image being visually inspected.
<b>Preparation for Use</b>	<ul style="list-style-type: none"> <li>Install the battery into the back of the phone. Ensure the back cover is closed tightly.</li> <li>Charge the battery before using the device for the first time.               <ol style="list-style-type: none"> <li>Connecting the charger improperly may cause serious damage to the mobile phone.</li> <li>The charger should remain close to the electrical socket and easily accessible while charging.</li> <li>After charging the device, ensure that the multipurpose jack cover is closed tightly.</li> </ol> </li> <li>Accessing the manual on the mobile phone, open the web browser and select the bookmark for the user manual from the bookmarks list. Also find the user manual at <a href="http://www.samsung.com">www.samsung.com</a>.</li> </ul>
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>For Wi-Fi access, contact your IT department for the network name and password to be able to connect to Wi-Fi in order to use the phone.</li> <li>Tap on Apps on the main home screen.</li> <li>Tap on Settings, tap the Wi-Fi icon and select from the list of names from the Wi-Fi networks screen to connect with the facility/department Wi-Fi.</li> </ol> <p><b>To download the magnifier for camera:</b></p> <ol style="list-style-type: none"> <li>Tap on the Apps icon, tap on the Play Store app.</li> <li>At the top of the page where it says 'Google Play', type in "magnifier for camera".</li> <li>Scroll down the list of apps to 'Camera Zoom-Zoom Enhancer' and tap to download app onto the phone. (This is a free app).</li> </ol>

	<ol style="list-style-type: none"> <li>4. After the installation is complete, the Camera Zoom app will appear on the phone's screen.</li> <li>5. Attach the phone to the phone adapter on the Fiberscope and follow the Fiberscope and LED Light Source IFU.</li> </ol>
<b>Interpretation of Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	<p>To view the full safety information, tap <b>Apps-Settings-About device-Legal information-Safety information</b>.</p> <ul style="list-style-type: none"> <li>● Use manufacturer-approved batteries, chargers, accessories, and supplies.</li> <li>● Do not expose the mobile phone to physical impact or damage.</li> <li>● Prevent the multipurpose jack and battery terminals from contacting conductive elements, such as metal or liquids.</li> <li>● Do not store the device in very hot or very cold areas. It is recommended to use the mobile phone at temperatures from 5°C (41°F) to 35°C (95°F).</li> <li>● Do not store the near or in heaters, microwaves, cooking equipment, or high pressure containers.</li> <li>● Do not touch the charger or the device with wet hands while the mobile phone is charging.</li> <li>● Do not use the mobile phone outdoor during a thunderstorm.</li> <li>● Turn off the mobile phone in potentially explosive environments.</li> <li>● Comply with all safety warnings and regulations regarding mobile phone usage while operating a vehicle.</li> <li>● If the mobile phone is cracked, broken, or becomes very hot, stop using the device immediately and take it to a Samsung Service Centre.</li> <li>● Do not poke eyes or other body parts with the device; do not insert device into the mouth.</li> <li>● If the mobile phone has camera flash or light, do not use it close to the eyes of people or animals.</li> <li>● To prevent possible hearing damage when using headset, do not listen to sound at high volume for long periods.</li> </ul>
<b>Disposal</b>	<ul style="list-style-type: none"> <li>● Charger, headset, and USB cable should not be disposed of with other household waste. This EEE is compliant with RoHS (Restriction of Hazardous Substances).</li> <li>● Battery, manual, or packaging should not be disposed of with other household items.</li> </ul> <p>To view more information about device and battery disposal, tap <b>Apps-Settings-About device-Legal information-Safety information</b>.</p>

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	<ul style="list-style-type: none"> <li>● Use pre-moistened sanitizing wipes or wipe the phone with a non-linting wipe and small amount of liquid hand sanitizer.</li> <li>● Use a dry microfiber cloth to clean fingerprints and/or smudges off the screen. Do not use any abrasive materials as this will scratch the screen.</li> </ul>
<b>Cleaning – Automated</b>	
<b>Disinfection</b>	
<b>Drying</b>	
<b>Maintenance, Inspection, and Testing</b>	<p><b>NOTE:</b> The mobile phone may be damaged if water or dust enters the phone. Follow these tips carefully to prevent damage to the mobile phone and to maintain the water-and dust-resistant performance of the mobile phone.</p> <ul style="list-style-type: none"> <li>● Do not immerse the device in water deeper than 1m and keep it submerged for more than 30 minutes.</li> <li>● Do not expose the device to water moving with force (ocean waves or waterfalls).</li> <li>● Do not open the device's covers when the mobile phone is in water or in very humid places (swimming pools or bathrooms).</li> <li>● Do not open the back cover using wet hands or when the mobile phone is wet.</li> </ul>

	<ul style="list-style-type: none"> <li>• The rubber seal fitted to the back cover is an important component of the mobile phone. Take care when opening and closing the back cover to avoid damaging the rubber seal. Also, ensure that the rubber seal is free from debris, such as sand or dust to prevent damage to the mobile phone.</li> <li>• Do not expose the mobile phone to salt water, ionized water, or soapy water.</li> <li>• The back and multipurpose jack cover may be loosened if the mobile phone is dropped or receives an impact. Ensure that all the covers are properly aligned and tightly closed.</li> <li>• If the device has been immersed in water or the microphone or the speaker is wet, sound may not be heard clearly during a call. Ensure that the microphone or the speaker is clean and dry by wiping it with a dry cloth.</li> <li>• If the device gets wet, dry it thoroughly with a clean, soft cloth.</li> <li>• The water-resistant design of the mobile phone causes it to vibrate slightly in certain conditions. Vibrations in these conditions, such as when the volume level is high, are normal and do not affect the device's performance.</li> <li>• The touch screen and other features may not work properly if the mobile phone is used in water or in other liquids.</li> </ul>
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	
<b>Storage</b>	
<b>Additional Information</b>	
<b>Related Healthmark Products</b>	
<b>Other Product Support Documents</b>	ProFormance™ Brochure, ProFormance™ Price List
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> hmark.com

2021-02-17 Suzanne Latta