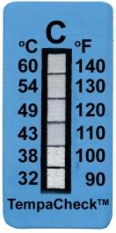
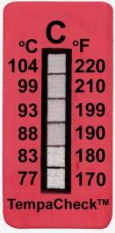


Instructions for Use: TempaChek™ Irreversible Thermometers

| | |
|--|---|
| Brand Name of Product | TempaChek™-90 & TempaChek™-170 Irreversible Thermometers |
| Generic Name of Product | Irreversible thermometers |
| Product Code Number(s) | Temp-90, Temp-170 |
| Purpose of Product | To monitor key temperatures during the wash cycle of an automated instrument washer. <ul style="list-style-type: none"> • Temp-90 – monitoring of the cold water (pre-wash) rinse. • Temp-170 – monitoring of the thermal disinfection rinse. |
| Range of Applications for Product | Automated instrument washers |
| Key specifications of product | <ul style="list-style-type: none"> • Irreversible thermometer – records the highest temperature achieved during exposure. • No residue adhesive allows placement on a clean, dry, smooth flat surface. • Easy to interpret color change from silver to black. • Graduated temperature scale to record range of temperatures. • Single-use. • Precision to +/-1°C. |

| Shipping & Storage | |
|---|---|
| Shipping Conditions & Requirements | Temp-90 – do not allow ship temperature to exceed 85°F. Temp-170 – do not allow ship temperature to exceed 165°F. |
| Storage Conditions | Keep in cool, dry place out of direct sunlight. Temp-90 – do not allow temperature to exceed 85°F. Temp-170 – do not allow temperature to exceed 165°F. |
| Packaging Conditions | 10 per pack |
| Shelf Life | |

| Instructions for Using Product | |
|---------------------------------------|---|
| Description of Use (s) | For monitoring key temperatures during the wash cycle of an automated instrument washer. |
| Preparation for Use | Be sure surface for adherence is dry, clean and smooth. |
| Diagrams (drawings, pictures): | <div style="display: flex; justify-content: space-around; align-items: center;"> <div style="text-align: center;">  <p><i>TempaChek™-90</i></p> </div> <div style="text-align: center;">  <p><i>TempaChek™-170</i></p> </div> </div> |
| Steps for Use of Product | <ul style="list-style-type: none"> • TempaChek™-90 <ol style="list-style-type: none"> 1. Use one (1) TempaChek™-90 per washer. 2. Peel thermometer from release paper. 3. Apply to any clean, dry flat surface ensuring that the indicator has adhered to the surface. 4. TempaChek™-90 should be read during the cold water rinse (pre-wash) stage. 5. Record results on log sheet. 6. Temperature should not exceed 100°F. Immediately report any result that exceeds this temperature. • TempaChek™-170 <ol style="list-style-type: none"> 1. Use one (1) TempaChek™-170 on each level of the instrument rack. 2. Peel thermometer from release paper. 3. Apply to any clean, dry flat surface ensuring that the indicator has adhered to the surface. 4. TempaChek™-170 should be read after the thermal disinfection stage and before the drying stage. 5. Record results on log sheet. |

| | |
|--|--|
| | 6. Report any deviation from targeted temperature. |
| Interpretation of Results | |
| Contraindications of Test Results | |
| Documentation | |
| Special Warnings and Cautions | <ol style="list-style-type: none"> 1. Peel off the thermometer(s) at the end of the wash cycle. 2. Surfaces may be hot, so take caution and wear appropriate safety gear. 3. Allowing the thermometer to stay on the surface for an extended period and for repeated cycles may make it very difficult to remove and adhesive residue may remain. 4. If adhesive residue remains on the surface, may be removed with acetone or isopropyl alcohol solvent. 5. Apply TempaChek™-90 and TempaChek™-170 labels only to a flat surface. Do not apply product to a round surface (the wash rack) as it may increase the risk of peeling and falling off during the wash cycle. |
| Disposal | May be disposed of in regular trash. |

| Reprocessing Instructions | |
|--|--|
| Point of use: | |
| Preparation for decontamination: | |
| Disassembly Instructions: | |
| Cleaning – Manual: | |
| Cleaning – Automated: | |
| Disinfection: | |
| Drying: | |
| Maintenance, inspection, and testing: | |
| Reassembly Instructions: | |
| Packaging: | |
| Sterilization: | |
| Storage: | |
| Additional Information: | |

| | |
|--|---|
| Related Healthmark Products | |
| Other Product Support Documents | ProFormance Brochure, ProFormance Price List |
| Reference Documents | |
| Customer Service contact: | Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com |

2018-02-14 Ralph J Basile