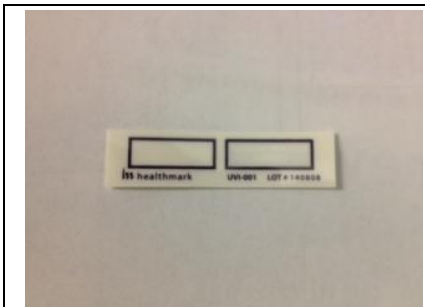



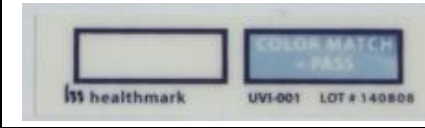

healthmark
INDUSTRIES CO.
health care products
800-521-6224
www.hmark.com

Instructions for Use: UV Irreversible Indicator

Brand Name of Product	UV Irreversible Indicator
Generic Name of Product	UV Indicator Test Strips
Product Code Number(s)	UVI-001
Purpose of Product	To test the indicate the efficacy of a UV disinfection system.
Range of Applications for Product	Systems that utilize ultra violet light at 254nm to disinfect surfaces
Key specifications of product	<ul style="list-style-type: none"> • Easy to interpret color change in the presence of UV light at 254nm during disinfection procedures. • No residue adhesive backing makes it possible to apply labels to various surfaces and remove without clean-up after use.

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	Keep covered and away from any light.
Packaging Conditions	
Shelf Life	

Instructions for Using Product	
Description of Use (s)	
Preparation for Use	
Diagrams (drawings, pictures):	<div style="display: flex; justify-content: space-around;"> <div style="text-align: center;">  <p>Fig 1 Indicator Strip Before Use</p> </div> <div style="text-align: center;">  <p>Fig 2 Comparison Key</p> </div> </div>

	 
	<p style="text-align: center;">Fig 3 Fail Result</p> <p style="text-align: center;">Fig 4 Pass Result</p>
Steps for Use of Product	<ol style="list-style-type: none"> 1. If used in a UV cabinet, may be placed on an open shelf without removing backing. 2. If used in a whole room UV disinfection procedure or similar application, peel adhesive off of the back of the indicator test strip and place on the testing surface. Test strip should be white as shown in Fig 1 before use. 3. Run UV disinfection cycle according to the UV system manufacturer's IFU. 4. Compare the right box of the test strip to the color on the comparison key shown in Fig 2. 5. If the color in the test strip is close in color or darker than the comparison key it is a pass result (Fig 4). If the color is noticeably lighter, it is a failed result (Fig 3). 6. Document result. 7. In the case of a failed test, consult the UV system manufacturer for possible causes and remedies.
Interpretation of Results	<ul style="list-style-type: none"> • If the color in the test strip is the close in color or darker than the comparison key it is a pass result (Fig 4). • If the color is noticeably lighter, it is a failed result (Fig 3).
Contraindications of Test Results	
Documentation	Document result according to facilities policy and procedure
Special Warnings and Cautions	Keep away from direct light until use.
Disposal	UV indicators are for single use. Adhesive easily peels off of surface. Dispose with normal trash.

Reprocessing Instructions	
Point of use:	
Preparation for decontamination:	
Disassembly Instructions:	
Cleaning – Manual:	
Cleaning – Automated:	
Disinfection:	
Drying:	
Maintenance, inspection, and testing:	
Reassembly Instructions:	
Packaging:	

Sterilization:	
Storage:	Keep covered and away from any light.
Additional Information:	

Related Healthmark Products	
Other Product Support Documents	ProFormance Brochure, ProFormance Price List
Reference Documents	
Customer Service contact:	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2014-08-27 Matt Smith