

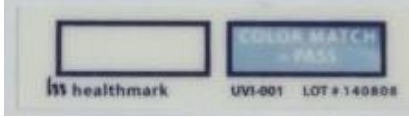
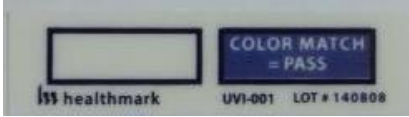


Instructions for Use: UV Irreversible Indicator

Brand Name of Product	UV Irreversible Indicator
Generic Name of Product	UV Indicator Test Strips
Product Code Number(s)	UVI-001
Intended Use	To indicate exposure to UV-C light.
Range of Applications for Product	Systems that utilize ultra-violet light at 254nm.
Key Specifications of Product	<ul style="list-style-type: none"> • Easy to interpret color change in the presence of UV light at 254nm. • “No residue” adhesive makes it possible to apply labels to various surfaces and remove without adhesive residue after use.

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	Keep covered and away from any light.
Packaging Contents	
Shelf Life	

Instructions for Using Product	
Description of Use(s)	
Preparation for Use	
Diagrams (drawings, pictures)	<div style="display: flex; justify-content: space-around;"> <div style="text-align: center;">  <p>Fig. 1 Indicator Strip Before Use</p> </div> <div style="text-align: center;">  <p>Fig. 2 Comparison Key</p> </div> </div> <div style="display: flex; justify-content: space-around; margin-top: 20px;"> <div style="text-align: center;">  <p>Fig. 3 Fail Result</p> </div> <div style="text-align: center;">  <p>Fig. 4 Pass Result</p> </div> </div>
Steps for Use of Product	<ol style="list-style-type: none"> 1. If used in a UV cabinet, may be placed on an open shelf, without removing backing. 2. If used in a whole room UV disinfection procedure or similar application, peel adhesive off of the back of the indicator test strip and place on the testing surface. Test strip should be white as shown in Fig 1 before use. 3. Run UV disinfection cycle according to the UV system manufacturer’s IFU. 4. Compare the right box of the test strip to the color on the comparison key shown in Fig 2. 5. If the color in the test strip is close in color or darker than the comparison key it is a pass result (Fig 4). If the color is noticeably lighter, it is a failed result (Fig 3). 6. Document result. 7. In the case of a failed test, consult the UV system manufacturer for possible causes and remedies.
Interpretation of Test Results	<ul style="list-style-type: none"> • If the color in the test strip is close in color or darker than the comparison key, it is a pass result. Fig. 4 • If the color is noticeably lighter, it is a failed result. Fig. 3
Contraindications of Test Results	
Documentation	Document results according to facility’s policy and procedure
Special Warnings and Cautions	Keep away from direct light until use.

Disposal	<ul style="list-style-type: none"> • UV indicators are single-use. • Dispose with normal trash.
-----------------	---

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	
Storage	Keep covered and away from any light.
Additional Information	
Related Healthmark Products	
Other Product Support Documents	ProFormance™ Brochure, ProFormance™ Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2020-08-05 Suzanne Latta