



ProFormanceQA™

Your Central Sterile Quality
Assurance and Infection Control
Documentation Solution

Staff Training

Logging in...

Step 1: Go to www.blacktiemedical.com and click 'LOG IN' in the upper right hand corner.

Step 2: Enter your staff username and password and click 'Login'

BlackTie - Inviting Formality... To The Healthcare Supply Chain - Windows Internet Explorer

https://www.blacktiemedical.com/Login.aspx

BlackTie
Inviting Formality...To The Healthcare Supply Chain

HIPAA Compliant, encrypted communication. Always.

HOME ABOUT US CONTACT US LOG IN

BILL-ONLY MANAGEMENT IMPLANT TRACKING CONTRACTS & DOCUMENTS QUALITY MANAGEMENT

Login

Username: staffuser

Password: [masked]

Login

Forgot [Username](#) or [Password](#)

LOOP PURCHASE TRANSACTION MANAGEMENT

LINK UTILIZATION MANAGEMENT

LOCK CONTRACT PORTFOLIO MANAGEMENT

LOOK VALUE MANAGEMENT

USE OF DATA COPYRIGHT CONTACT US TERMS OF USE BLACKTIE PR & MEDIA PRIVACY

Internet 100%

Conducting Inspections...

- Once you have logged in you will be brought to this page.
- Before you begin, choose your facility, your name on the inspector list, and your location.

The screenshot shows a web browser window displaying the BlackTie ProFormanceQA™ application. The browser's address bar shows the URL <https://www.blacktiemedical.com/WasherStaff.aspx>. The application header includes the BlackTie logo, navigation icons for Central Sterile, and the healthmark logo. The user is logged in as 'Hospital Staff' at Lourdes Medical Center. The main navigation menu includes 'Bill-Only Management', 'Implant Tracking', 'Contracts & Documents', and 'ProFormanceQA™'. The 'Conduct Inspections' section is active, showing a form with three dropdown menus: 'Facility: Lourdes Medical Center - Detroit', 'Inspector: Kirk, Alex', and 'Location: Basement'. A red oval highlights these dropdown menus. To the right of the form is a button labeled 'Conduct Unscheduled Test'. Below the form, there are two columns of test results: 'Tests Currently Due' and 'As Needed'. The 'Tests Currently Due' column lists various washer types (e.g., Ultrasonic Cleaner, Cart Washer, Tunnel Washer, Lumen Washer, Multi Washer, SpotCheck) with colored circular indicators. The 'As Needed' column lists tests like EndoCheck, HemoCheck, and ChannelCheck, also with colored circular indicators. The browser's status bar at the bottom shows 'Done' and 'Internet'.

Conducting Inspections...

- On the left side of the screen you can view all currently due tests.
- On the right side of the screen you can view all tests that are on an 'as needed' basis.
- To log data, simply click on the colored bubble next to the equipment you wish to enter test data for. This will bring you to the log sheet.

The screenshot shows a web browser window titled "Washer Tests - Windows Internet Explorer" with the URL "https://www.blacktiemedical.com/WasherStaff.aspx". The page header includes logos for BlackTie, Central Sterile, and healthmark INDUSTRIES CO. The navigation menu includes "Bill-Only Management", "Implant Tracking", "Contracts & Documents", and "ProFormanceQA™". The main content area is titled "Conduct Inspections" and features filters for Facility (Lourdes Medical Center - Detroit), Inspector (Kirk, Alex), and Location (Basement). A "Conduct Unscheduled Test" button is also present. The interface is divided into two columns: "Tests Currently Due" and "As Needed".

Tests Currently Due

- Ultrasonic Cleaner Basement Sonic 1
- Ultrasonic Cleaner Basement Sonic 2
- Cart Washer Basement Cart Washer
- Tunnel Washer Basement Tunnel Washer 1
- Lumen Washer Basement SurgiSonic1
- Multi Washer Basement Multi Rack Washer 1
- Multi Washer Basement Multi Rack Washer 2
- SpotCheck Basement
- SpotCheck Basement

As Needed

- EndoCheck Basement
- HemoCheck Basement
- ChannelCheck Basement

Conducting Inspections...

-You can also conduct an unscheduled test (if the equipment you have tested is not shown in the 'Tests Currently Due' section).

-To do so, simply click the 'Conduct Unscheduled Test' Button.

The screenshot shows a web browser window titled "Washer Tests - Windows Internet Explorer" with the URL "https://www.blacktiemedical.com/WasherStaff.aspx". The page header includes logos for BlackTie, Central Sterile, and healthmark INDUSTRIES CO. The navigation menu contains "Bill-Only Management", "Implant Tracking", "Contracts & Documents", and "ProFormanceQA™". The main content area is titled "Conduct Inspections" and features a form with the following fields: Facility: "Lourdes Medical Center - Detroit", Inspector: "Kirk, Alex", and Location: "Basement". A button labeled "Conduct Unscheduled Test" is circled in red. Below the form, there are two columns of test results: "Tests Currently Due" and "As Needed".

Tests Currently Due

- Ultrasonic Cleaner Basement Sonic 1
- Ultrasonic Cleaner Basement Sonic 2
- Cart Washer Basement Cart Washer
- Tunnel Washer Basement Tunnel Washer 1
- Lumen Washer Basement SurgiSonic1
- Multi Washer Basement Multi Rack Washer 1
- Multi Washer Basement Multi Rack Washer 2
- SpotCheck Basement
- SpotCheck Basement

As Needed

- EndoCheck Basement
- HemoCheck Basement
- ChannelCheck Basement

Unscheduled Tests and Back Dating...

- Once you have clicked the 'Conduct Unscheduled Test' button, this menu will pop up momentarily.
- Simply choose the proper equipment, the date of the test, and click 'Save'.
- Once you have clicked 'Save' you will be brought to the log sheet.

Washer Tests - Windows Internet Explorer

https://www.blacktiemedical.com/WasherStaff.aspx?iID=213&fID=Lourdes+Medical+Center++Detroit&iID=42&dID=

File Edit View Favorites Tools Help

Washer Tests

BlackTie Inviting Formality...To The Healthcare Supply Chain

Central Sterile

healthmark INDUSTRIES CO health care products

Home | About Us | Contact | Log Out
Lourdes Medical Center
User: Hospital Staff

Bill-Only Management Implant Tracking Contracts & Documents ProFormanceQA™

Conduct Inspections

Conduct Inspections

Facility: Lourdes Medical Center - Detroit Inspector: Kirk, Alex Location: Basement Conduct Unscheduled Test Go To Sterilizers

Tests Currently Due As Needed

Unscheduled Test

*Test: Water Quality - Basement

*Test Date: 04/06/2009

April, 2009

Su	Mo	Tu	We	Th	Fr	Sa
29	30	31	1	2	3	4
5	6	7	8	9	10	11
12	13	14	15	16	17	18
19	20	21	22	23	24	25
26	27	28	29	30	1	2
3	4	5	6	7	8	9

Today: April 6, 2009

Ultrasonic Cleaner Basement Sonic 1

Ultrasonic Cleaner Basement Sonic 2

Cart Washer Basement Cart Washer

Tunnel Washer Basement Tunnel Washer 1

Lumen Washer Basement SurgiSonic1

Multi Washer Basement Multi Rack Washer 1

Multi Washer Basement Multi Rack Washer 2

SpotCheck Basement

SpotCheck Basement

Done Internet 100%

The Log Sheet...

- Before you begin, verify that your name is selected as the inspector.
- Also, make sure that the date and location are correct.

Tunnel Washer Test - Windows Internet Explorer

https://www.blacktiemedical.com/TunnelWasherTest.aspx?tID=14&JID=213&FID=Lourdes%20Medical%20Center%20-%20Detroit&JID=42

File Edit View Favorites Tools Help

Tunnel Washer Test

BlackTie Inviting Formality...To The Healthcare Supply Chain

Central Sterilization

healthmark INDUSTRIES CO health care products

Home | About Us | Contact | Log Out
Lourdes Medical Center
User: Hospital Staff

Bill-Only Management Implant Tracking Contracts & Documents **ProFormanceQA™**

Conduct Inspections

Tunnel Washer Test

Inspector: Kirk, Alex Date: 4/6/2009 Location: Basement

Water Temperature

C	F
60	140
54	130
49	120
43	110
38	100
32	90

TempaChek -90: Note:

Water Temperature

C	F
104	220
99	210
93	199

TempaChek Note:

Done Internet 100%

Filling out the Log Sheet...

- As you scroll down through the log sheet you will see all of the test results that you must enter.
- Notice, there is a section for notes next to each field.
- To enter data, simply click on the proper bubble in each field.
- Once you have selected the proper result in each category, click 'Save' in the bottom right corner of the page.

The screenshot shows a web browser window titled "Tunnel Washer Test - Windows Internet Explorer". The address bar shows the URL: <https://www.blacktiemedical.com/TunnelWasherTest.aspx?tID=16&JID=&FID=&JID=>. The browser's menu bar includes File, Edit, View, Favorites, Tools, and Help. The page content is organized into several sections:

- TempaCheck -90:** A vertical column of radio buttons for temperature selection. The options are: C 60, F 140; C 54, F 130; C 49, F 120; C 43, F 110; C 38, F 100; C 32, F 90. A "Note" text box is to the right, which is circled in red.
- Water Temperature**: A vertical column of radio buttons for water temperature selection. The options are: C 104, F 220; C 99, F 210; C 93, F 199; C 88, F 190; C 83, F 180; C 77, F 174. A "Note" text box is to the right.
- SonoCheck**: Three radio buttons labeled "Blue", "Green", and "Yellow". A "Note" text box is to the right.
- SonoCheck Placement**: A 3x3 grid of radio buttons. A "Note" text box is to the right.
- TOSI Result**: A horizontal row of radio buttons labeled 0, 1, 2, 3, 4, 5. A "Note" text box is to the right.

The browser's status bar at the bottom shows "Done" and "Internet" with a 75% zoom level.

Congratulations!

You have completed
ProFormanceQA Staff Training.

If you have any questions or comments,
please contact Alex Kirk at Healthmark Industries.

Phone: (800)521-6224

Email: alex@hmark.com

Thank You!