



healthmark
INDUSTRIES CO.
health care products
800-521-6224
www.hmark.com

Instructions for Use: Healthmark Watermark™

Brand Name of Product	Healthmark Watermark™
Generic Name of Product	Water level guide and temperature gauge
Product Code Number(s)	TEMP-WMK
Purpose of Product	To indicate the appropriate water volume and temperature in a liquid bath.
Range of Applications for Product	When submerging surgical instruments in a liquid bath
Key specifications of product	

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	
Shelf Life	

Instructions for Using Product	
Description of Use (s)	To gauge the appropriate fill line and temperature of the water.
Preparation for Use	<ol style="list-style-type: none"> 1. Be sure the surface for adherence is dry. 2. Read instructions from detergent manufacture and the cleaning equipment or sink. 3. Record the volume of water in the white box on the Healthmark Watermark
Diagrams (drawings, pictures):	
Steps for Use of Product	<ol style="list-style-type: none"> 1. Determine the appropriate fill line in the tank where the instruments are to be submerged. 2. Remove adhesive label on the back. 3. Place watermark on water fill line. 4. Add cleaning solution according the instructions of the detergent manufacture. 5. Staff should then be instructed as to the amount of detergent to add to the bath, in accordance with the detergent manufacturer's IFU.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions	
Point of use:	
Preparation for decontamination:	
Disassembly Instructions:	
Cleaning – Manual:	
Cleaning – Automated:	
Disinfection:	
Drying:	
Maintenance, inspection, and testing:	<ul style="list-style-type: none"> • Replace the Healthmark Watermark when adhesive back wears off and Watermark begins to peel. • After long term use, the liquid crystals may lose their elasticity and the color change will lose its vibrancy. Change the Watermark should this happen.
Reassembly Instructions:	
Packaging:	
Sterilization:	
Storage:	
Additional Information:	

Related Healthmark Products	
Other Product Support Documents	ProFormance Price List, ProFormance Brochure
Reference Documents	
Customer Service contact:	Healthmark Industries Company, Inc

	33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com
--	---

2012-11-21 msmith