

Instructions for Use: Brush Storage Rack & Brush Soak Tray

Brand Name of Product	Brush Storage Rack & Brush Soak Tray
Generic Name of Product	Brush Storage Rack & Brush Soak Tray
Product Code Number(s)	BR001, BRS03
Intended Use	The storage rack holds the brushes to allow the bristles to be submerged in cleaning solution in the soaking tray.
Range of Applications for Product	To hold the brushes and allow bristles to be submerged in the soaking tray.
Key Specifications of Product	<ul style="list-style-type: none"> • Clear Plexiglas • Two holes for mounting • BR001- Brush Storage Rack <ul style="list-style-type: none"> ○ 10 slots per rack • BRS03- Brush Soaking Tray

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Contents	
Shelf Life	

Instructions for Using Product	
Description of Use(s)	To hold the brushes and allow bristles to be submerged into the soaking tray.
Preparation for Use	<ol style="list-style-type: none"> 1. Mount the storage rack to the wall surface with two screws. 2. Mount the brush soak tray(s) below the storage rack at different heights to accommodate both shorter and longer brushes stored on the same rack. 3. Use two screws per soaking tray to be mounted to the wall surface.
Diagrams (drawings, pictures)	
Steps for Use of Product	<ol style="list-style-type: none"> 1. Place the bristles of the brush end into the soaking tray. 2. Place the eyelet of the brush on the hook to hang on the storage rack.
Interpretation of Test Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	<ul style="list-style-type: none"> • Remove the soaking tray from the wall and empty the cleaning solution. • Clean the storage rack and soaking tray with neutral pH detergent and warm water.
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	
Storage	
Additional Information	
Related Healthmark Products	

Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2021-03-05 Suzanne Latta