



Instructions for Use: Elevator Mechanism Brush IFU

Brand Name of Product	Elevator Mechanism Brush
Generic Name of Product	Brush
Product Code Number(s)	EMB-002
Intended Use	To clean around the forceps elevator area of duodenoscopes and other endoscopes with an elevator mechanism such as ultrasonic endoscopes
Range of Applications for Product	Endoscope with elevators
Key Specifications of Product	0.4 mm brush and handle

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	
Shelf Life	

Instructions for Using Product	
Description of Use(s)	Use the brush to clean both sides of the forceps elevator mechanism and recesses of duodenoscopes and other endoscopes with elevator mechanisms.
Preparation for Use	Remove from packaging.
Diagrams (drawings, pictures)	
Steps for Use of Product	<ul style="list-style-type: none"> • Brush in accordance with the endoscope manufacturer's IFU¹: <ol style="list-style-type: none"> a. Lower the forceps elevator and straighten the bending section of the endoscope. b. While keeping the distal end of the endoscope immersed in detergent solution, brush the forceps elevator and forceps elevator recess as follows: <ol style="list-style-type: none"> i. Brush the guidewire-locking groove with three strokes. ii. Brush both grooves with three strokes each. c. Gently clean the bristles of the brush with your fingertips in the detergent solution. d. Raise the forceps elevator. e. Brush the back side of the forceps elevator three times while keeping the distal end of the endoscope immersed in detergent solution. f. Gently clean the bristles of the brush with your fingertips in the detergent solution. g. Slowly insert the brush head into the groove under the forceps elevator until the end of the brush touches the wall behind the elevator. h. While keeping the distal end of the endoscope immersed in the detergent solution, rotate the inserted brush three revolutions while ensuring the end of the brush is contacting the wall behind the forceps elevator. i. Pull the brush out slowly and gently clean the bristles in the detergent solution with your fingertips. j. Repeat steps (g) through (i). k. Inspect whether there is debris on the forceps elevator and in the forceps elevator recess while raising and lowering the forceps elevator, and repeat brushing and/or flushing the forceps elevator and the forceps elevator recess until no debris is observed upon the inspection. <p>Reference:</p> <ol style="list-style-type: none"> 1. Olympus Instructions for Single Use Soft Bristle MAJ-1888. 2015.

Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	These brushes are intended for use as cleaning tools only. They are not for use in patient care.
Disposal	For use on a single device only. When finished using the brush, it should be treated as biohazard and disposed of in accordance to your facility's guidelines.

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	
Storage	
Additional Information	

Related Healthmark Products	
Other Product Support Documents	ProSys™ Price List, ProSys™ Brochure
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2020-08-06 Matt Smith