



	<ol style="list-style-type: none"> <li>b. Use the brush and the cleaning solution of choice to gently cleanse the valve openings.</li> <li>c. Avoid twisting or torque of the brush handle. The brush handle will prevent the brush from being pushed too deeply in the valve openings.</li> <li>d. Use the brush and the cleaning solution of choice to clean the control head and the control head dials.</li> </ol> <ol style="list-style-type: none"> <li>3. Channel Cleaning Brushes: <ol style="list-style-type: none"> <li>a. Select the size brush appropriate for the channel to be cleaned.</li> <li>b. Always brush channel while submerged in the cleaning solution.</li> <li>c. Use short strokes advancing the brush completely through the channel until it emerges from the distal end.</li> <li>d. Carefully pull the entire cleaning brush completely out of the distal end.</li> <li>e. Inspect whether there is debris on the bristles. If debris is observed, submerge the bristle end of the brush in the cleaning solution using gloved fingertips to remove the debris.</li> <li>f. Repeat steps (a) through (e) until no debris is observed on the bristles.</li> <li>g. Dispose of the brush after use.</li> </ol> </li> </ol>
<b>Interpretation of Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	<ul style="list-style-type: none"> <li>● Endoscope cleaning brushes are for single-use only.</li> <li>● Endoscope cleaning brushes cannot be reused or reprocessed.</li> <li>● Consult and follow closely the IFU from the endoscope manufacturer and the cleaning agent manufacturer.</li> <li>● Select a brush based upon the diameter and length of the channel chosen. Do not force a brush into a channel to avoid possible damage to the endoscope.</li> </ul>
<b>Disposal</b>	Treat used endoscope brushes as a biohazard and dispose of in accordance with facility policy of contaminated items.

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	
<b>Cleaning – Automated</b>	
<b>Disinfection</b>	
<b>Drying</b>	
<b>Maintenance, Inspection, and Testing</b>	
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	
<b>Storage</b>	
<b>Additional Information</b>	

<b>Related Healthmark Products</b>	ProSys™ Brushes
<b>Other Product Support Documents</b>	ProSys™ Brochure, ProSys™ Price List
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>