

Flexible Endoscopes General Diameter Guide  
for Endoscopy Brushes

SCOPE TYPE	INSERTION TUBE OUTER DIAMETER	WORKING LENGTH	INSTRUMENT CHANNEL INTERNAL	HEALTHMARK ENDO BRUSHES	INTENDED CHANNEL DIAMETER	VALVE BRUSHES & ELEVATOR MECHANISM BRUSH	OLYMPUS COLOR GUIDE
<b>ADULTS</b>							
GASTROSCOPE	9.0 mm - 11.4 mm	103-105 cm	2.8 mm - 3.8 mm	CC-230-600	2.6-4.9 mm	00711610 - 12mm, 3770 - 6.4 & 1mm	Green, Yellow, Orange
DUODENOSCOPE*	10.8 mm - 12.5 mm	125 cm	3.2 mm - 4.2 mm	CC-230-600	2.6-4.9 mm	00711610 - 12mm, 3770 - 6.4 & 1mm, EMB-001 -0.4mm	Green, Yellow, Orange
COLONOSCOPE	12.9 mm - 13.7 mm	133-168 cm	3.7 mm - 4.2 mm	CC-230-600	2.6-4.9 mm	00711610 - 12mm, 3770 - 6.4 & 1mm	Green, Yellow, Orange
SIGMOIDOSCOPE	12.8 mm - 13.2 mm	70-73 cm	3.7 mm - 4.2 mm	CC-230-600	2.6-4.9 mm	00711610 - 12mm, 3770 - 6.4 & 1mm	Green, Yellow, Orange
ENTEROSCOPE	10.5 mm - 11.7 mm	220-250 cm	2.8 mm - 3.8 mm	CC-230-600	2.6-4.9mm	00711610 - 12mm, 3770 - 6.4 & 1mm	Green, Yellow, Orange
BRONCHOSCOPE	5.7 mm - 6.0 mm	55-60 cm	2.0 mm - 2.8 mm	CC-230-400, CC-230-600	1.7-2.5 mm, 2.6-4.9 mm	00711610 - 12mm, 3770 - 6.4 & 1mm	Blue, Purple, Green, Yellow, Orange
URETEROSCOPE	2.8 mm - 3.3 mm	67-70 cm	1.2 mm	CC-200-200	1.1-1.6mm	00711610 - 12mm, 3770 - 6.4 & 1mm	White
CYSTOSCOPE	5.4 mm - 5.5 mm	38 cm	2.2 mm - 2.4 mm	CC-230-400	1.7-2.5mm	00711610 - 12mm, 3770 - 6.4 & 1mm	Blue, Purple
RHINO-LARYNGOSCOPE	2.6 mm - 4.9 mm	30-36.5 cm	N/A - 2.0	CC-230-400	1.7-2.5mm	00711610 - 12mm, 3770 - 6.4 & 1mm	Blue, Purple
LAPARO-THORACOSCOPE	7.0 mm	27 cm	2.8 mm	CC-230-600	2.6-4.9 mm	00711610 - 12mm, 3770 - 6.4 & 1mm	Green, Yellow, Orange
MOBILE AIRWAY SCOPE	4.1 mm - 5.2 mm	60 cm	1.5 mm - 2.6 mm	CC-230-400, CC-230-600	1.7-2.5 mm, 2.6-4.9 mm	00711610 - 12mm, 3770 - 6.4 & 1mm	Blue, Purple, Green, Yellow, Orange
CHOLEDOCHOSCOPE	2.8 mm - 5.2 mm	38-70 cm	1.2 mm - 2.2 mm	CC-200-200, CC-230-400	1.1-1.6 mm, 1.7-2.5 mm	00711610 - 12mm, 3770 - 6.4 & 1mm	White, Blue, Purple
EUS Linear Scope*	11.8 mm - 12.6 mm	125 cm	2.8 mm - 3.7 mm	CC-230-600	2.6-4.9 mm	00711610 - 12mm, 3770 - 6.4 & 1mm, EMB-001 -0.4mm	Green, Yellow, Orange
EUS Radial Scope*	10.5 mm- 11.8 mm	125 cm	2.2 mm	CC-230-400	1.7-2.5mm	00711610 - 12mm, 3770 - 6.4 & 1mm	Blue, Purple
EBUS Scopes	6.2 mm	60 cm	2.2 mm	CC-230-400	1.7-2.5mm	00711610 - 12mm, 3770 - 6.4 & 1mm	Blue, Purple
<b>PEDIATRICS</b>							
GASTROSCOPE	5.9 mm - 6.0 mm	103-105 cm	2.0 mm	CC-230-400	1.7-2.5 mm	00711610 - 12mm, 3770 - 6.4 & 1mm	Blue, Purple
COLONOSCOPE	11.5 mm - 11.6 mm	168-170 cm	3.2 mm - 3.8 mm	CC-230-600	2.6-4.9 mm	00711610 - 12mm, 3770 - 6.4 & 1mm	Green, Yellow, Orange
BRONCHOSCOPE	4.4 mm - 5.1 mm	60 cm	2.0 mm	CC-230-400	1.7-2.5 mm	00711610 - 12mm, 3770 - 6.4 & 1mm	Blue, Purple

\*Must be fully manually cleaned prior to going into the ASP EvoTech for disinfection. Includes Olympus® duodenoscopes (TJF, JF, PJF), dual channel scopes (2T), ultrasound scopes (UC and UMs).

Refer to ASP for more information.