

Lumen Sizer User Guide

Item # JP007

The Lumen Sizer helps to determine the internal diameter of a lumen opening to select the appropriate brush for cleaning the internal surface of lumens.

1. To find the correct size of a lumen opening, place the opening of the lumen over the raised posts of lumen sizes until the correct size is determined. **(Fig.1)**

Figure
1



2. Once the correct size is determined, use this number to select the corresponding size brush for cleaning the internal surface of the lumen.
3. The following range of sizes for the Lumen Sizer are: 1.35 mm, 1.75 mm, 2 mm, 2.45 mm, 2.75 mm, 3 mm, 3.45 mm, 3.75 mm, 4 mm, 4.45 mm, 4.75 mm, 5 mm, 5.45 mm, 5.75 mm, 6 mm, 6.45 mm, 7 mm, 7.75 mm.
4. Insert the inspection tool into the lumen to check for cleanliness of the internal surface.
5. If there is visible debris found on the Lumen Sizer after sizing the lumen, send the Lumen Sizer back to decontam for reprocessing.
 - a. To manually clean the Lumen Sizer, wipe down with disinfecting wipes.
 - b. For cleaning & disinfection, use a neutral pH detergent. Place Lumen Sizer in a (secure) tray and place on washer rack. Set the washer-disinfector on the “instrument cycle” and run the cycle.