



Instructions for Use: Swab Sticks

Brand Name of Product	Swab Sticks
Generic Name of Product	Swab Sticks
Product Code Number(s)	TX714A
Intended Use	Intended for manually cleaning items. Can also be used as sampling swab for Hemocheck™ and ProCheck®.
Range of Applications for Product	N/A
Key Specifications Product	Can be used for cleaning and for sampling.

Shipping & Storage

Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product

Description of Use(s)	To allow for safe cleaning of sensitive items or to collect samples for reagent tests.
Preparation for Use	<ul style="list-style-type: none"> • Be sure to follow the device manufacturer’s IFU. • If utilizing a cleaning agent follow the detergent manufacturer’s IFU for proper preparation of the cleaning solution.
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<ul style="list-style-type: none"> • When using as a testing swab for cleaning verification products, follow the IFU of the appropriate ProFormance™ product. • These swab sticks are too long to allow for capping the ProFormance™ reagent tests. To use, swish within the reagent with the cap off. Leave the stick immersed in the reagent solution for the time specified by the reagent IFU. Remove the stick and inspect the swab part of the indicated color change. • When used for cleaning instruments: <ul style="list-style-type: none"> ○ Moisten with cleaning solution (if applicable). ○ Swab instrument to remove soils and debris. ○ Dispose after use on a single instrument.
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	<ul style="list-style-type: none"> • Consult the IFU of the device manufacturer to ensure full compliance with their instructions. • Pay special attention to area(s) of the item that must be free from moisture. • Pay special attention to chemical compatibility of the medical device and areas of the medical device with the cleaning agent you select.
Disposal	Treat as biohazard and discard in compliance with facility guidelines.

Reprocessing Instructions

Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A

Sterilization	N/A
Storage	N/A
Additional Information	N/A

Related Healthmark Products	N/A
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2021-05-24 MSmith