



Instructions for Use: Clamps & Spreaders

Brand Name of Product	Clamps & Spreaders
Generic Name of Product	Clamps & Spreaders
Product Code Number(s)	20411, 20408, 20406, 20412, 20407
Purpose of Product	To keep surgical instruments in an open position while being cleaned.
Range of Applications for Product	<ul style="list-style-type: none"> • Clamps ensure that joints of jaws of the bone forceps remain open. • Spreaders are for instruments with ring handles that cannot be dismantled. They can be cleaned more efficiently in the washer-disinfector. The jaws remain open during the cleaning process, thus assuring thorough cleaning.
Key specifications of product	<ul style="list-style-type: none"> • Stainless Steel • Silicone Covering

Shipping & Storage

Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	
Shelf Life	

Instructions for Using Product

Description of Use (s)	When instruments are closed, they are not cleaned properly. Water and detergent cannot reach all of the surfaces of the instrument. This leads to an increase chance of contamination.
Preparation for Use	Make sure you select the appropriate size clamp before using on instrument.
Diagrams (drawings, pictures):	
Steps for Use of Product	<ul style="list-style-type: none"> • Open surgical instrument to the appropriate width for cleaning. • Insert ends of clamps or spreaders through instrument loops. This will help keep medical device in an open position for cleaning. • Continue normal cleaning procedure according to instrument manufacture guidelines.
Interpretation of Results	
Contraindications of Test Results	
Documentation	AAMI, AORN, device manufactures and others strongly recommend that instruments be kept in an open position when cleaned, to insure maximum exposure to the cleaning solution.
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions

Point of use:	
Preparation for decontamination:	
Disassembly Instructions:	
Cleaning – Manual:	
Cleaning – Automated:	Can be cleaned through the automatic washer with surgical instruments.
Disinfection:	
Drying:	
Maintenance, inspection, and testing:	
Reassembly Instructions:	
Packaging:	
Sterilization:	Materially compatible for Steam and EO sterilization.
Storage:	
Additional Information:	

Related Healthmark Products	
Other Product Support Documents	Prosys Brochure, Prosys Price List
Reference Documents	

Customer Service contact:

Healthmark Industries Company, Inc
33671 Doreka
Fraser, MI 48026
1-586-774-7600
healthmark@hmark.com
hmark.com

2017-08-09 msmith