

## Instructions for Use: Clamps & Spreaders

|  |   |
|--|---|
| <b>Brand Name of Product</b>             | Clamps & Spreaders  |
| <b>Generic Name of Product</b>           | Clamps & Spreaders  |
| <b>Product Code Number(s)</b>            | 20411, 20408, 20406, 20412, 20407   |
| <b>Purpose of Product</b>                | To keep the item in an open position while being cleaned.   |
| <b>Range of Applications for Product</b> | <ul style="list-style-type: none"> <li>• Clamps ensure that joints of jaws of the bone forceps remain open.</li> <li>• Spreaders are for items with ring handles that cannot be dismantled. They can be cleaned more efficiently in the washer-disinfector. The jaws remain open during the cleaning process, thus assuring thorough cleaning.</li> </ul> |
| <b>Key Specifications of Product</b>     | <ul style="list-style-type: none"> <li>• Stainless Steel</li> <li>• Silicone Covering</li> </ul>  |

| Shipping & Storage                            |  |
|---|--|
| <b>Shipping Conditions &amp; Requirements</b> |  |
| <b>Storage Conditions</b>                     |  |
| <b>Packaging Conditions</b>                   |  |
| <b>Shelf Life</b>                             |  |

| Instructions for Using Product           |  |
|--|--|
| <b>Description of Use (s)</b>            | When items are closed, they are not cleaned properly. Water and detergent cannot reach all the surface of the item. This leads to an increase chance of contamination.   |
| <b>Preparation for Use</b>               | Make sure you select the appropriate size clamp before using on the item.  |
| <b>Diagrams (drawings, pictures)</b>     |  |
| <b>Steps for Use of Product</b>          | <ul style="list-style-type: none"> <li>• Open the item to the appropriate width for cleaning.</li> <li>• Insert the ends of clamps or spreaders through the loops and the rings of the handle. This will help keep items in an open position for cleaning.</li> <li>• Continue normal cleaning procedure according to manufacture guidelines.</li> </ul> |
| <b>Interpretation of Results</b>         |  |
| <b>Contraindications of Test Results</b> |  |
| <b>Documentation</b>                     | AAMI, AORN, device manufacturers and others strongly recommend that item be kept in an open position when cleaned, to insure maximum exposure to the cleaning solution.  |
| <b>Special Warnings and Cautions</b>     |  |
| <b>Disposal</b>                          |  |

| Reprocessing Instructions                   |   |
|---|---|
| <b>Point of Use</b>                         |   |
| <b>Preparation for Decontamination</b>      |   |
| <b>Disassembly Instructions</b>             |   |
| <b>Cleaning – Manual</b>                    |   |
| <b>Cleaning – Automated</b>                 | Can be cleaned through the automatic washer with other items.   |
| <b>Disinfection</b>                         |   |
| <b>Drying</b>                               |   |
| <b>Maintenance, Inspection, and Testing</b> |   |
| <b>Reassembly Instructions</b>              |   |
| <b>Packaging</b>                            |   |
| <b>Sterilization</b>                        | <ul style="list-style-type: none"> <li>• Steam sterilization for 4 minutes at 132°C and for 3 minutes at 135°C</li> <li>• EtO sterilization.</li> </ul> |
| <b>Storage</b>                              |   |
| <b>Additional Information</b>               |   |

|  |  |
|--|--|
| <b>Related Healthmark Products</b>     |  |
| <b>Other Product Support Documents</b> | ProSys™ Brochure, ProSys™ Price List               |
| <b>Reference Documents</b>             |  |
| <b>Customer Service Contact</b>        | Healthmark Industries Company, Inc<br>33671 Doreka |

|  |   |
|--|---|
|  | Fraser, MI 48026<br>1-586-774-7600<br><a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a><br>hmark.com |
|--|---|

2018-11-30 msmith