

## Instructions for Use: Clamps & Spreaders

<b>Brand Name of Product</b>	Clamps & Spreaders
<b>Generic Name of Product</b>	Clamps & Spreaders
<b>Product Code Number(s)</b>	20411, 20408, 20406, 20412, 20407, T0200
<b>Purpose of Product</b>	To keep the item in an open position while being cleaned.
<b>Range of Applications for Product</b>	<ul style="list-style-type: none"> <li>• Clamps ensure that joints of jaws of the bone forceps remain open.</li> <li>• Spreaders are for items with ring handles that cannot be dismantled. They can be cleaned more efficiently in the washer-disinfector. The jaws remain open during the cleaning process, thus assuring thorough cleaning.</li> </ul>
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• Stainless Steel</li> <li>• Silicone Covering</li> </ul>

### Shipping & Storage

<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Contents</b>	
<b>Shelf Life</b>	

### Instructions for Using Product

<b>Description of Use (s)</b>	When items are closed, they are not cleaned properly. Water and detergent cannot reach all the surface of the item. This leads to an increase chance of contamination.
<b>Preparation for Use</b>	Make sure you select the appropriate size clamp before using on the item.
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<ul style="list-style-type: none"> <li>• Open the item to the appropriate width for cleaning.</li> <li>• Insert the ends of clamps or spreaders through the loops and the rings of the handle. This will help keep items in an open position for cleaning.</li> <li>• Continue normal cleaning procedure according to manufacture guidelines.</li> </ul>
<b>Interpretation of Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	AAMI, AORN, device manufacturers and others strongly recommend that item be kept in an open position when cleaned, to insure maximum exposure to the cleaning solution.
<b>Special Warnings and Cautions</b>	
<b>Disposal</b>	

### Reprocessing Instructions

<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	
<b>Cleaning – Automated</b>	Can be cleaned through the automatic washer with other items.
<b>Disinfection</b>	
<b>Drying</b>	
<b>Maintenance, Inspection, and Testing</b>	
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	<ul style="list-style-type: none"> <li>• Steam sterilization 4 minutes at 132°C for and for 3 minutes at 135°C</li> <li>• EtO sterilization.</li> </ul>
<b>Storage</b>	
<b>Additional Information</b>	

<b>Related Healthmark Products</b>	
<b>Other Product Support Documents</b>	ProSys™ Brochure, ProSys™ Price List
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc 18600 Malyn Blvd.

	Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> hmark.com
--	---

2020-08-28 msmith