



Instructions for Use: Crescent Style Stringer

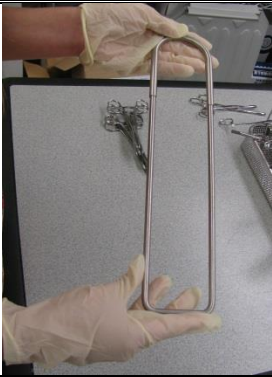
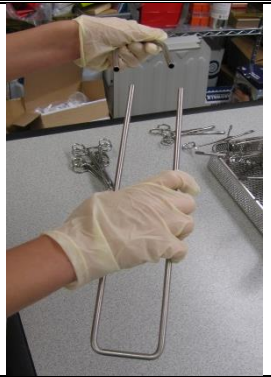
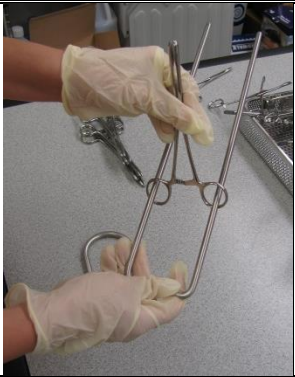

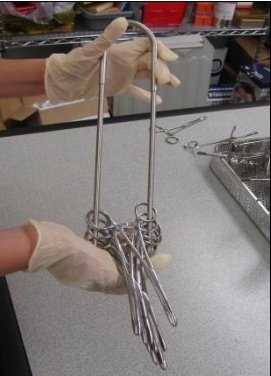
Brand Name of Product	Crescent Style Stringers
Generic Name of Product	Crescent Style Instrument Stringers
Product Code Number(s)	ST2.64C, ST2.84C, ST2.104C, ST2.124C, ST2.144C, ST2.164C, ST2.184C, ST3-64C, ST3-84C, ST3-104C, ST3-124C, ST3-144C, ST3-164C, ST3-184C
Purpose of Product	This product is used to expand instruments to better expose hard to reach areas during cleaning and sterilization.
Range of Applications for Product	
Key specifications of product	<ul style="list-style-type: none"> Manufactured of stainless steel

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	
Shelf Life	

Instructions for Using Product

Description of Use (s)	Keeps instruments maximally exposed for hard to reach areas during cleaning and sterilization.
-------------------------------	--

Preparation for Use	
----------------------------	--

Diagrams (drawings, pictures):			
	1	2	3
			
	4	5	

Steps for Use of Product	<ol style="list-style-type: none"> Place one hand at each end of the stringer. Using the hand over the crescent end of the stringer, pull to separate the two pieces. String handled instruments in set assembly order onto the stringer by placing one ring on each of the stringer arms. Continue until all instruments are loaded onto the stringer. Lock the stringer by reattaching the crescent end. You may need to gently pull one of the arms outward to thread into the crescent end.
---------------------------------	--

Interpretation of Results	
----------------------------------	--

Contraindications of Test Results	
--	--

Documentation	
----------------------	--

Special Warnings and Cautions	
Disposal	

Reprocessing Instructions	
Point of use:	
Preparation for decontamination:	
Disassembly Instructions:	
Cleaning – Manual:	<ul style="list-style-type: none"> • Before initial use of the Stringer, cleaning is required. <ul style="list-style-type: none"> ○ Manually wash stringer(s) with a neutral pH detergent and warm water. Expand and collapse the adjustable assembly area and actuate locking mechanisms during the washing process. ○ Place the stringer(s) in ultrasonic cleaner for effective cleaning ○ Place stringer(s) in automated washer system for effective cleaning • After initial use the Stringer is adequately cleaned during the cleaning of the instruments.
Cleaning – Automated:	<ol style="list-style-type: none"> 1. See instructions under “Steps for Use of Product” for directions on how to string instruments. 2. When using for the cleaning of instruments in an automated washer, open stringer as wide as possible to maximize exposure of the instrument to the cleaning solution. Consult the instrument manufacturer’s IFU for specific instructions.
Disinfection:	
Drying:	
Maintenance, inspection, and testing:	Apply instrument milk to sliding cross bar mechanism as needed to ensure smooth action.
Reassembly Instructions:	
Packaging:	
Sterilization:	<ol style="list-style-type: none"> 1. See instructions under “Steps for Use of Product” for directions on how to string instruments. 2. When using for sterilization of instruments open to the optimal width for use in the instrument set. Consult the instrument manufacturer’s IFU for specific instructions. 3. Stringers are materially compatible with steam and EO sterilization.
Storage:	
Additional Information:	

Related Healthmark Products	Stringers
Other Product Support Documents	Prosys Price List, Prosys Brochure
Reference Documents	
Customer Service contact:	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2014-08-28 troom