

## Instructions for Use: EZ Lok Double-Hinge Bar

|  |  |
|--|--|
| <b>Brand Name of Product</b>             | EZ Lok Double-Hinge Bar  |
| <b>Generic Name of Product</b>           | Instrument stringers   |
| <b>Product Code Number(s)</b>            | PSIS-6275, PSIS-8275, PSIS-10275, PSIS-12275   |
| <b>Purpose of Product</b>                | This product is used to expand instruments to better expose hard to reach areas during cleaning and sterilization.   |
| <b>Range of Applications for Product</b> | N/A  |
| <b>Key Specifications of Product</b>     | <ul style="list-style-type: none"> <li>• Manufactured stainless steel</li> <li>• Double-hinge bar connects in the middle with male/female closure</li> </ul> |

| <b>Shipping &amp; Storage</b>                 |     |
|---|-----|
| <b>Shipping Conditions &amp; Requirements</b> | N/A |
| <b>Storage Conditions</b>                     | N/A |
| <b>Packaging Contents</b>                     | N/A |
| <b>Shelf Life</b>                             | N/A |

| <b>Instructions for Using Product</b>    |   |
|--|---|
| <b>Description of Use (s)</b>            | Keeps instruments maximally exposed for hard-to-reach areas during cleaning and sterilization.  |
| <b>Preparation for Use</b>               | N/A   |
| <b>Diagrams (drawings, pictures)</b>     | N/A   |
| <b>Steps for Use of Product</b>          | <ol style="list-style-type: none"> <li>1. Unlock the stringer by pulling outward gently on both sides of the stringer and lifting the hinges upward.</li> <li>2. String handled instruments with rings in set assembly order onto the stringer by placing one ring on each of the stringer arms.</li> <li>3. Continue until all instruments are loaded onto the stringer.</li> <li>4. Lower the hinges back down toward the hinge, gently, and fit the pin into the socket to lock the instruments in place.</li> </ol> |
| <b>Interpretation of Results</b>         | N/A   |
| <b>Contraindications of Test Results</b> | N/A   |
| <b>Documentation</b>                     | N/A   |
| <b>Special Warnings and Cautions</b>     | N/A   |
| <b>Disposal</b>                          | N/A   |

| <b>Reprocessing Instructions</b>            |   |
|---|---|
| <b>Point of Use</b>                         | N/A   |
| <b>Preparation for Decontamination</b>      | N/A   |
| <b>Disassembly Instructions</b>             |   |
| <b>Cleaning – Manual</b>                    | <ul style="list-style-type: none"> <li>• Before initial use of the EZ Lok Stringer, cleaning is required.               <ul style="list-style-type: none"> <li>○ Manually wash stringer(s) with a neutral pH detergent and warm water.</li> <li>○ Place the stringer(s) in ultrasonic cleaner for effective cleaning.</li> <li>○ Put stringer(s) in automated washer system for effective cleaning.</li> </ul> </li> <li>• After initial use, the stringer is adequately cleaned during instrument cleaning.</li> </ul> |
| <b>Cleaning – Automated</b>                 | See <i>Steps for Use of Product</i> for directions on how to string instruments.  |
| <b>Disinfection</b>                         | N/A   |
| <b>Drying</b>                               |   |
| <b>Maintenance, Inspection, and Testing</b> | N/A   |
| <b>Reassembly Instructions</b>              | N/A   |
| <b>Packaging</b>                            | N/A   |
| <b>Sterilization</b>                        | <ul style="list-style-type: none"> <li>• See <i>Steps for Use of Product</i> for directions on how to string instruments.</li> <li>• When using for sterilization of instruments, open the EZ Lok Stringer to the optimal width for use in the instrument set. (NOTE: Consult the instrument manufacturer's IFU for specific instructions.)</li> </ul>  |

|  |   |
|--|---|
|  | <ul style="list-style-type: none"> <li>Stringers are materially compatible with steam and EO sterilization.</li> </ul>  |
| <b>Storage</b>                         | N/A   |
| <b>Additional Information</b>          | N/A   |
| <b>Related Healthmark Products</b>     | Stringers   |
| <b>Other Product Support Documents</b> | ProSys™ Price List, ProSys™ Brochure  |
| <b>Reference Documents</b>             | N/A   |
| <b>Customer Service Contact</b>        | Healthmark Industries Company, Inc<br>18600 Malyn Blvd.<br>Fraser, MI 48026<br>1-586-774-7600<br><a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a><br><a href="http://hmark.com">hmark.com</a> |

2021-10-05 troom