

Instructions for Use: EZ Lok Double-Hinge Bar

Brand Name of Product	EZ Lok Double-Hinge Bar
Generic Name of Product	Instrument stringers
Product Code Number(s)	PSIS-6275, PSIS-8275, PSIS-10275, PSIS-12275, HE8-0018
Purpose of Product	This product is used to expand instruments to better expose hard to reach areas during cleaning and sterilization.
Range of Applications for Product	N/A
Key Specifications of Product	<ul style="list-style-type: none"> • Manufactured stainless steel • Double-hinge bar connects in the middle with male/female closure

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use (s)	Keeps instruments maximally exposed for hard-to-reach areas during cleaning and sterilization.
Preparation for Use	N/A
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<ol style="list-style-type: none"> 1. Unlock the stringer by pulling outward gently on both sides of the stringer and lifting the hinges upward. 2. String handled instruments with rings in set assembly order onto the stringer by placing one ring on each of the stringer arms. 3. Continue until all instruments are loaded onto the stringer. 4. Lower the hinges back down toward the hinge, gently, and fit the pin into the socket to lock the instruments in place.
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	N/A
Disposal	N/A

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	
Cleaning – Manual	<ul style="list-style-type: none"> • Before initial use of the EZ Lok Stringer, cleaning is required. <ul style="list-style-type: none"> ○ Manually wash stringer(s) with a neutral pH detergent and warm water. ○ Place the stringer(s) in ultrasonic cleaner for effective cleaning. ○ Put stringer(s) in automated washer system for effective cleaning. • After initial use, the stringer is adequately cleaned during instrument cleaning.
Cleaning – Automated	See <i>Steps for Use of Product</i> for directions on how to string instruments.
Disinfection	N/A
Drying	
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	<ul style="list-style-type: none"> • See <i>Steps for Use of Product</i> for directions on how to string instruments. • When using for sterilization of instruments, open the EZ Lok Stringer to the optimal width for use in the instrument set. (NOTE: Consult the instrument manufacturer’s IFU for specific instructions.)

	<ul style="list-style-type: none"> Stringers are materially compatible with steam and EO sterilization.
Storage	N/A
Additional Information	N/A

Related Healthmark Products	Stringers
Other Product Support Documents	ProSys™ Price List, ProSys™ Brochure
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com