



**Instructions for Use: Center Latch Instrument Stringers**

<b>Brand Name of Product</b>	Center Latch Instrument Stringers
<b>Generic Name of Product</b>	Instrument stringers
<b>Product Code Number(s)</b>	STCL-10, STCL -6
<b>Purpose of Product</b>	This product is used to expand instruments to better expose hard to reach areas during cleaning and sterilization.
<b>Range of Applications for Product</b>	
<b>Key specifications of product</b>	<ul style="list-style-type: none"> <li>Manufactured of stainless steel</li> </ul>

**Shipping & Storage**

<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Shelf Life</b>	

**Instructions for Using Product**

<b>Description of Use (s)</b>	Keeps instruments maximally exposed for hard to reach areas during cleaning and sterilization.
<b>Preparation for Use</b>	
<b>Diagrams (drawings, pictures):</b>	
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>Unlock the stringer by gently pushing the right side of the latch up, while holding the left side of the latch in place.</li> <li>Once the latch is unlocked, extend both sides of the latch so they are completely extended. String ring handled instruments in set assembly order onto stringer by placing one ring on each of the stringer arms. Continue until all instruments are loaded onto stringer.</li> <li>Lock latch by closing the latch into a secure locking position.</li> </ol>
<b>Interpretation of Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	
<b>Disposal</b>	

**Reprocessing Instructions**

<b>Point of use:</b>	
<b>Preparation for decontamination:</b>	
<b>Disassembly Instructions:</b>	
<b>Cleaning – Manual:</b>	<ul style="list-style-type: none"> <li>Before initial use of the stringer, cleaning is required. <ul style="list-style-type: none"> <li>Manually wash stringer(s) with a neutral ph detergent and warm water.</li> </ul> </li> </ul>
<b>Cleaning – Automated:</b>	
<b>Disinfection:</b>	
<b>Drying:</b>	
<b>Maintenance, inspection, and testing:</b>	
<b>Reassembly Instructions:</b>	
<b>Packaging:</b>	

<b>Sterilization:</b>	Stringers are materially compatible with steam and EO sterilization.
<b>Storage:</b>	
<b>Additional Information:</b>	

<b>Related Healthmark Products</b>	Stringers
<b>Other Product Support Documents</b>	Prosys Price List, Prosys Brochure
<b>Reference Documents</b>	
<b>Customer Service contact:</b>	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>

2014-03-11 msmith