


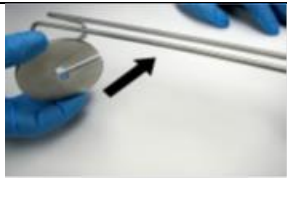
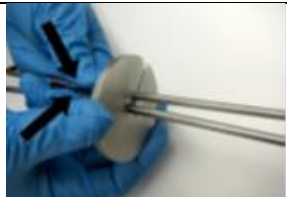

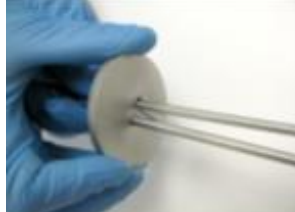



Brand Name of Product	Wash Cradle
Generic Name of Product	Instrument wash cradle
Product Code Number(s)	IWC-001
Purpose of Product	The Wash Cradle is designed to hold surgical instruments in the open position to promote effective cleaning in the automated washer with a minimum amount of staff effort.
Range of Applications for Product	For hinged instruments that need to be held open during the washing process.
Key specifications of Product	<ul style="list-style-type: none"> • 11.5” Length, 2” Width, 3.75” Height • Manufactured of stainless steel • Adjustable rack length

Shipping & Storage

Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	
Shelf Life	

Instructions for Using Product

Description of Use (s)	The design will increase effectiveness of the cleaning process of ring handled/hinged instruments by presenting them in an open and upright position. This positioning helps expose the most difficult to clean locations to washing (water & detergents) and instrument lubricants.		
Preparation for Use	Methods to Load: 1) Instruments are loose in basket.		
Diagrams (drawings, pictures)	Assembly		
			
	1	2	3
			
	4	5	6


Loading



7



8

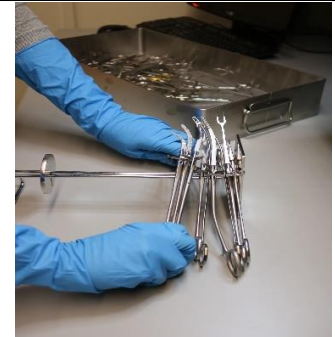


9



10



11

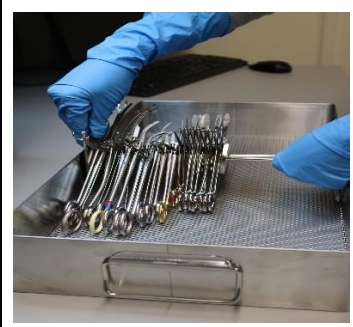

12



13



14

Steps for Use of Product

Assembly

1. Cradle and key lock.
2. Install key lock by sliding open slit over wire guides.
3. Pinch wire guides together.
4. Turn the key lock until wire guides snap into place.
5. Slide key lock back.
6. Ready to load.

Loading

7. Be sure to open instruments to maximize exposure for cleaning.
8. Place each instrument onto the cradle rack.
9. Continue loading instruments onto the rack until all instruments are in place.
10. Slide key lock to place to secure instruments.
11. Pick up the wash cradle to place into basket.
12. Place the wash cradle into the basket.
13. Be sure that the wash cradle is secure in the basket. It can be used in various styles of baskets as long as the instruments are secure.
14. Instruments are ready to be washed.

Interpretation of Results

Contraindications of Test Results

Documentation

Special Warnings and Cautions	
Disposal	

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	
Storage	
Additional Information	

Related Healthmark Products	Washer baskets, Nic Nac Washer Baskets & Balls
Other Product Support Documents	Prosys™ Brochure, Prosys™ Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2018-05-16 msmith