



Instructions for Use: Pin Locking Instrument Stringers

Brand Name of Product	Pin Locking Instrument Stringers
Generic Name of Product	Instrument stringers
Product Code Number(s)	ST-6, ST55-8
Purpose of Product	This product is used to expand instruments to better expose hard to reach areas during cleaning and sterilization.
Range of Applications for Product	
Key specifications of product	<ul style="list-style-type: none"> Manufactured of stainless steel

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Shelf Life	

Instructions for Using Product	
Description of Use (s)	Keeps instruments maximally exposed for hard to reach areas during cleaning and sterilization.
Preparation for Use	
Diagrams (drawings, pictures):	<div style="display: flex; justify-content: space-around; margin-top: 5px;"> 1 2 3 </div>
Steps for Use of Product	<ol style="list-style-type: none"> Unlock the stringer by gently pulling the lever arm outwards, while pulling the locked latch upwards. String ring handled instruments in set assembly order onto stringer by placing one ring on each of the stringer arms. Continue until all instruments are loaded onto stringer. Lock latch by closing the latch into a secure locking position.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions	
Point of use:	
Preparation for decontamination:	
Disassembly Instructions:	
Cleaning – Manual:	<ul style="list-style-type: none"> Before initial use of the stringer, cleaning is required. <ul style="list-style-type: none"> Manually wash stringer(s) with a neutral pH detergent and warm water.
Cleaning – Automated:	
Disinfection:	
Drying:	
Maintenance, inspection, and testing:	
Reassembly Instructions:	
Packaging:	
Sterilization:	Stringers are materially compatible with steam and EO sterilization.
Storage:	

Additional Information:	
Related Healthmark Products	Stringers
Other Product Support Documents	Prosys Price List, Prosys Brochure
Reference Documents	
Customer Service contact:	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2014-03-11 msmith