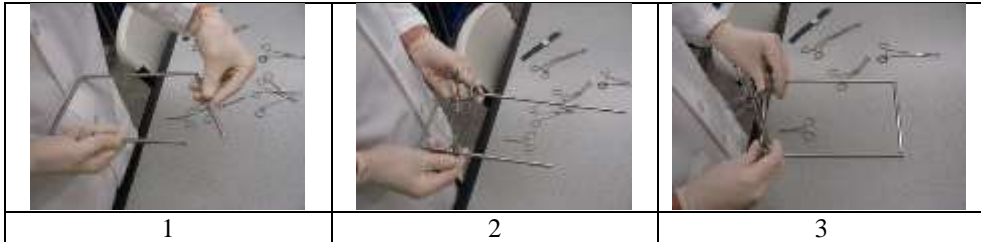


**Instructions for Use: Pin Locking Instrument Stringers**

<b>Brand Name of Product</b>	Pin Locking Instrument Stringers
<b>Generic Name of Product</b>	Instrument stringers
<b>Product Code Number(s)</b>	ST-6, ST-10, ST-12, ST55-8
<b>Purpose of Product</b>	This product is used to expand instruments to better expose hard to reach areas during cleaning and sterilization.
<b>Range of Applications for Product</b>	
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>Manufactured of stainless steel</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Contents</b>	
<b>Shelf Life</b>	

<b>Instructions for Using Product</b>	
<b>Description of Use (s)</b>	Keeps instruments maximally exposed for hard to reach areas during cleaning and sterilization.
<b>Preparation for Use</b>	
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>Unlock the stringer by gently pulling the lever arm outwards, while pulling the locked latch upwards.</li> <li>String ring handled instruments in set assembly order onto stringer by placing one ring on each of the stringer arms. Continue until all instruments are loaded onto stringer.</li> <li>Lock latch by closing the latch into a secure locking position.</li> </ol>
<b>Interpretation of Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	
<b>Disposal</b>	

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	<ul style="list-style-type: none"> <li>Before initial use of the stringer, cleaning is required.               <ul style="list-style-type: none"> <li>Manually wash stringer(s) with a neutral pH detergent and warm water.</li> </ul> </li> </ul>
<b>Cleaning – Automated</b>	
<b>Disinfection</b>	
<b>Drying</b>	
<b>Maintenance, Inspection, and Testing</b>	
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	Stringers are materially compatible with steam and EO sterilization.
<b>Storage</b>	

<b>Additional Information</b>	
<b>Related Healthmark Products</b>	Stringers
<b>Other Product Support Documents</b>	ProSys™ Price List, ProSys™ Brochure
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>

2021-02-03 msmith