




Instructions for Use: Sauder Instrument Holder

Brand Name of Product	Sauder Instrument Holder
Generic Name of Product	Stringer
Product Code Number(s)	10-1780
Intended Use	Keeps ring handled instruments maximally exposed during sterilization.
Range of Applications for Product	Used for ring handled instruments.
Key Specifications of Product	Manufactured of stainless steel.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	Keeps ring handled instruments maximally exposed during sterilization.
Preparation for Use	N/A

Diagrams (drawings, pictures)	<div style="display: flex; justify-content: space-around;"> <div style="text-align: center;">  <p>Figure 1</p> </div> <div style="text-align: center;">  <p>Figure 2</p> </div> </div> <div style="text-align: center; margin-top: 20px;">  <p>Figure 3</p> </div>
--------------------------------------	--

Steps for Use of Product	<ol style="list-style-type: none"> 1. Unhook both bars of the stringer. Fig. 1 2. Place the ring handled instruments in the open position. 3. The rings of the instrument are put through both bars of the stringer. Fig. 2 4. When finished placing the instruments through the bars, push the bars back underneath the hooks to secure them in place. Fig. 3 5. The instruments should be facing in a forward fashion.
---------------------------------	--

Interpretation of Test Results	N/A
Contraindications of Test Results	N/A

Documentation	N/A
Special Warnings and Cautions	N/A
Disposal	N/A

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	Before initial use of the stringer, cleaning is required. Manually wash stringer(s) with a neutral pH detergent and warm water.
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	<ul style="list-style-type: none"> ● See instructions under “Steps for Use of Product” for directions on how to string instruments. ● When using for sterilization of instruments, open to the optimal width for use in the instrument set. Consult the instrument manufacturer’s IFU for specific instructions. ● Stringers are materially compatible with steam and EO sterilization. ● Steam sterilization at 132°C (270°F) for 4 minutes or 135°C (275°F) for 3 minutes. ● Maximum steam sterilization 135°C (275°F) for 30 minutes.
Storage	N/A
Additional Information	N/A
Related Healthmark Products	Stringers
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2021-04-22 Suzanne Latta