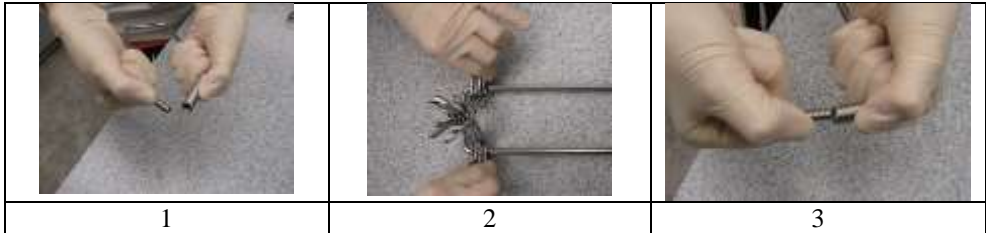


Instructions for Use: Spring Locking Instrument Stringers

Brand Name of Product	Spring Locking Instrument Stringers
Generic Name of Product	Instrument stringers
Product Code Number(s)	USPRS-10, USPRS-12, USPRS-14, USPRS-16, USPRS-6, USPRS-8
Purpose of Product	This product is used to expand instruments to better expose hard to reach areas during cleaning and sterilization.
Range of Applications for Product	
Key Specifications of Product	<ul style="list-style-type: none"> Manufactured of stainless steel

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Contents	
Shelf Life	

Instructions for Using Product	
Description of Use (s)	Keeps instruments maximally exposed for hard to reach areas during cleaning and sterilization.
Preparation for Use	
Diagrams (drawings, pictures)	 <div style="display: flex; justify-content: space-around; margin-top: 5px;"> 1 2 3 </div>
Steps for Use of Product	<ol style="list-style-type: none"> Unlock the stringer by gently pulling the smaller center piece of the spring out from inside the larger piece. Once the latch is unlocked, extend both sides of the latch so they are completely extended. String ring handled instruments in set assembly order onto stringer by placing one ring on each of the stringer arms. Continue until all instruments are loaded onto stringer. Lock latch by closing the latch into a secure locking position. The smaller end of the spring will insert inside the larger spring.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	<ul style="list-style-type: none"> Before initial use of the stringer, cleaning is required. <ul style="list-style-type: none"> Manually wash stringer(s) with a neutral ph detergent and warm water.
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	Stringers are materially compatible with steam and EO sterilization.

Storage	
Additional Information	
Related Healthmark Products	Stringers
Other Product Support Documents	ProSys™ Price List, ProSys™ Brochure
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2021-02-09 msmith