



Instructions for Use: Stainless Steel Washing Sleeve

Brand Name of Product	Stainless Steel Washing Sleeve
Generic Name of Product	Stainless Steel Washing Sleeve
Product Code Number(s)	07433, 07436, 07439, 07441
Intended Use:	Intended for effective transportation, washing and storage of delicate medical devices.
Range of Applications for Product	
Key specifications of product	<ul style="list-style-type: none"> • Perforated Stainless Steel • 2 silicone caps

Shipping & Storage

Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	
Shelf Life	

Instructions for Using Product

Description of Use (s)	Manufactured for effective transportation, washing and storage of delicate medical devices.
Preparation for Use	
Diagrams (drawings, pictures):	
Steps for Use of Product	<ul style="list-style-type: none"> • Place instrument inside washing sleeve • Secure caps to each end of the washing sleeve • Begin reprocessing in accordance to medical device manufacturer’s IFU.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions

Point of use:	
Preparation for decontamination:	
Disassembly Instructions:	
Cleaning – Manual:	Use cleaning solution approved for use with stainless steel.
Cleaning – Automated:	<ul style="list-style-type: none"> • Select the “Instrument Cycle” on the automated washer. • Use cleaning solution approved for use with stainless steel.
Disinfection:	
Drying:	
Maintenance, inspection, and testing:	
Reassembly Instructions:	
Packaging:	
Sterilization:	Steam Sterilization: <ul style="list-style-type: none"> • 135 °C / 275 °F / 3 min • 132 °C / 270 °F / 4 min
Storage:	
Additional Information:	

Related Healthmark Products	
Other Product Support Documents	ProSys Brochure, ProSys Price List
Reference Documents	
Customer Service contact:	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

