



Instructions for Use: Glove Box Dispenser-Single & Triple with Dividers

Brand Name of Product	Glove Box Dispenser
Generic Name of Product	Glove Box Dispenser
Product Code Number(s)	GBD-S1, GBD-T3
Intended Use	To hold one box or three boxes of extended-cuff gloves to be placed in all patient care areas requiring extended-cuff gloves.
Range of Applications for Product	Used in any area using hazardous chemicals: surgical areas, Sterile Processing areas,, oncology, pharmacy, and veterinary, or any area for the use of decontamination processes.
Key Specifications of Product	<ul style="list-style-type: none"> ● Holds one or three boxes of gloves ● Made of clear PETG Plastic- Copolyester resin ● Keyholes for wall mounting ● Dimensions of glove box dispenser <ul style="list-style-type: none"> ○ Single box dispenser <ul style="list-style-type: none"> - 5.45” W x 16.10” H x 2.95” D (13.8 cm x 40.9 cm x 7.5 cm) ○ Weighs 0.8 lbs. ○ Triple box dispenser with dividers <ul style="list-style-type: none"> - 15.70” W x 16.10” H x 2.70” D (39.9 cm x 40.9 cm x 6.9 cm) - 1.8 lbs.

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	
Shelf Life	

Instructions for Using Product	
Description of Use(s)	To hold one box or three boxes of gloves to be placed in all patient care areas.
Preparation for Use	<ol style="list-style-type: none"> 1. Remove the dispenser from packaging. 2. Securely mount to the wall by inserting the screws into the wall. 3. Place the dispenser onto the screws at the bottom. 4. Then place the dispenser with the two screw holders that are located on the back of the dispenser onto the screws to hold the dispenser in place.
Diagrams (drawings, pictures)	
Steps for Use of Product	<ul style="list-style-type: none"> ● Pull the perforated tab off of the box(es) of gloves. ● From the top of the dispenser, slide glove box down into the dispenser with the opening of the box facing forward. ● For the triple box dispenser, slide a glove box down into each of the dividers with the opening of the box facing forward. ● Pull gloves out of the box one at a time. ● Dispose of after single use.
Interpretation of Test Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	<ul style="list-style-type: none"> ● Use a soft clean with warm water, and mild soap. ● Durable enough to use disinfecting wipes.

Cleaning – Automated	
Disinfection	<ul style="list-style-type: none"> ● Acceptable agents: <ol style="list-style-type: none"> 1. Chlorine bleach 2. Isopropyl alcohol 3. Hydrogen peroxide 4. Quaternary ammonia wipes
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	
Storage	
Additional Information	
Related Healthmark Products	
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2019-05-02 Suzanne Latta

Rev. A