



Instructions for Use: Heat Mitts & Gloves

Brand Name of Product	Insulated Gauntlet Mitts, Seamless Autoclave Gloves and Terry Cloth Autoclave Gloves
Generic Name of Product	Heat Mitts & Gloves
Product Code Number(s)	75574, 75524, 73224, JB-2636, JB-422-5, JB-422-11
Purpose of Product	For removing hot trays and racks from autoclaves
Range of Applications for Product	Tray, racks and items which are hot that are being removed from the autoclave
Key specifications of product	<ul style="list-style-type: none"> Insulated Gauntlet Mitts includes protective insulation rated to 350°F. Seamless Autoclave Gloves are 100% cotton with exclusive high-density loop structure rated to 450°F. Terry Cloth Gloves are 100% cotton with gauntlet style cuffs rated to 250°F.

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	
Shelf Life	

Instructions for Using Product	
Description of Use (s)	
Preparation for Use	
Diagrams (drawings, pictures):	
Steps for Use of Product	<ol style="list-style-type: none"> 1. Insert hands completely into the mitts. 2. Be sure that the mitts are completely covering your hands for maximum protection.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions	
Point of use:	
Preparation for decontamination:	
Disassembly Instructions:	
Cleaning – Manual:	
Cleaning – Automated:	<ul style="list-style-type: none"> It is extremely important that you “DO NOT” use any bleach or dry cleaning solution when cleaning these products. <ol style="list-style-type: none"> 1) Rinse gloves/sleeves in cold water. 2) If presoaking use luke warm water to ensure good particle softening. Mild detergents or chemicals should be used to enhance the laundering process. 3) All products should be laundered using a mild soap and detergent. 4) Use a warm water wash with a temperature not to exceed 160°F (71°C). Wash time should not be longer than 10 to 15 minutes. 5) Rinse product in cold water. 6) Repeat above as often as needed to ensure a thoroughly clean product. It is important to use a high extraction speed to thoroughly remove all excess moisture.
Disinfection:	
Drying:	<ul style="list-style-type: none"> Tumbles dry all seamless knitted products in a temperature of not higher than 150°F (67°C) for a time of 10 to 15 minutes.

	<ul style="list-style-type: none"> • Tumble dry terry products in the same conditions as above but the time to dry may increase depending on quantity and weight of the products. Temperatures should not exceed 150°F (67°C).
Maintenance, inspection, and testing:	
Reassembly Instructions:	
Packaging:	
Sterilization:	
Storage:	
Additional Information:	

Related Healthmark Products	Seamless Autoclave Gloves, Terry Cloth Autoclave Gloves, Shark Tooth Heat Mitts
Other Product Support Documents	Prosys Brochure, Prosys Price List
Reference Documents	
Customer Service contact:	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2017-08-09 msmith