



33671 Doreka
Fraser, Michigan 48026
Telephone: (586) 491-2002
Fax: (586) 491-2112
www.HMARK.com

Laundering Instructions for All Jomac Products containing 100% Kevlar®, Cotton and blends of Polyester/Cotton or Kevlar.

The Jomac products are manufactured using state of the art machinery. The consistent quality of the product is ensured by multiple quality checks within the manufacturing process. The Jomac products are manufactured with Kevlar® (DuPont) and the highest quality of raw materials that is available. When laundering the gloves please follow the simple rules outlined below.

- It is extremely important that you “**DO NOT**” use any bleach or dry cleaning solution when cleaning these products.
- Rinse gloves/sleeves in cold water.
- If presoaking use luke warm water to ensure good particle softening. Mild detergents or chemicals should be used to enhance the laundering process.
- All products should be laundered using a mild soap and detergent.
- Use a warm water wash with a temperature not to exceed 160°F (71°C). Wash time should not be longer than 10 to 15 minutes.
- Rinse product in cold water.
- Repeat above as often as needed to ensure a thoroughly clean product.
- It is important to use a high extraction speed to thoroughly remove all excess moisture.
- Tumbles dry all seamless knitted products in a temperature of not higher than 150°F (67°C) for a time of 10 to 15 minutes.
- Tumble dry terry products in the same conditions as above but the time to dry may increase depending on quantity and weight of the products. Temperatures should not exceed 150°F (67°C).

Note: If there are special conditions that arise please consultant your Jomac specialist for additional information and recommendations to launder your Jomac products.