

Instructions for Use: Silicone Heat Glove

Brand Name of Product	Silicone Heat Glove
Generic Name of Product	Heat Glove
Product Code Number(s)	09484, 09368
Intended Use	For the safe handling of hot trays and racks from autoclaves.
Range of Applications for Product	Tray, racks, and items that are hot being removed from the autoclave.
Key Specifications of Product	<ul style="list-style-type: none"> Made of silicone. Protects from surface temperatures ranging from -40 °F to 464 °F.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	Stored at room temperature. (Do not expose to direct sunlight)
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	N/A
Preparation for Use	N/A
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<ol style="list-style-type: none"> Insert hands completely into the gloves. Ensure the gloves are securely covering your hands for maximum protection.
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	<ul style="list-style-type: none"> The gloves can withstand temperatures of 240 °C (464 °F). Gloves do not protect against chemicals.
Disposal	N/A

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	<ul style="list-style-type: none"> Detergent (pH < 10). Consult the detergent manufacturer’s IFU for the recommended concentration, exposure times and temperature. It is recommended to use demineralized water for the final rinse.
Cleaning – Automated	<ul style="list-style-type: none"> Acidic / neutral / alkaline < pH 10. Consult the detergent manufacturer’s IFU for the recommended concentration, exposure times, and temperature. Consult the IFU of the washer manufacturer for correct cycle settings. Do not clean in an ultrasonic cleaner.
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	<ul style="list-style-type: none"> Visually inspect gloves for usability and cleanliness. Damaged gloves no longer provide protection and must be replaced.
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	Compatible with steam sterilization: <ul style="list-style-type: none"> 132 °C (270 °F) for 4 minutes 135 °C (275 °F) for 3 minutes. Extended cycle: 135 °C (275 °F) for 30 minutes.
Storage	N/A
Additional Information	N/A

Related Healthmark Products	Seamless Autoclave Gloves, Terry Cloth Autoclave Gloves, Shark Tooth Heat Mitts
------------------------------------	---

Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2021-06-18 msmith