






Instructions for Use: All-in-One Microscope Camera

Brand Name of Product	iLoupe XL2 All-in-One Microscope Camera
Generic Name of Product	Microscope Camera
Product Code Number(s)	26700-520
Purpose of Product	Designed for professional optical inspection of surgical instruments and other equipment.
Range of Applications for Product	
Key specifications of product	<ul style="list-style-type: none"> • One of the lightest, most powerful, and compact mobile microscopes in the world • Palm-sized microscope that does not require any additional monitors to view subjects, giving freedom to take it anywhere you need magnification • Full color display allows for the viewing of images at high magnification, the ability to take photos, the ability to record videos, and the ability to display output on a PC • May also be used as a web camera • Four bright, built-in LEDs with intensity control allows for control of the illumination • The included software has powerful measurement function to measure length, perimeter, angle, and area of the object in view • 2” LCD video screen • Wide magnification range of 3X – 300X • Built-in 5 mega-pixel camera • Features: 180° rotated lens tube, SD card for storing and transferring images; easy-to-use touchpad controls, rechargeable lithium-ion battery, TV video output

Shipping & Storage

Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	
Shelf Life	

Instructions for Using Product

Description of Use (s)	Use for inspection of instruments and other equipment.
Preparation for Use	
Diagrams (drawings, pictures):	
Steps for Use of Product	<ol style="list-style-type: none"> 1) Press the power button located in the upper right-hand corner of the microscope until the screen lights up and images appear. 2) Use the high and low magnification control on the left-hand side of the microscope to adjust to achieve desired magnification/focus. 3) To take a snapshot of the image on the screen, press the snapshot button located on the left-hand side of the top of the microscope. An hourglass figure will appear and then your snapshot will appear on the screen. 4) To view photos, press the  button on the lower right-hand side of the microscope. Navigate through the photos using the left and right arrows of the touchpad. 5) To exit the saved photos, pressure the  button again. 6) The function menu can be accessed by pressing the home/trash can button. Navigate through the function menu with the arrows on the touchpad. Select the desired function by pressing the center of the touchpad.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions

Point of use:	
Preparation for decontamination:	
Disassembly Instructions:	
Cleaning – Manual:	
Cleaning – Automated:	
Disinfection:	
Drying:	
Maintenance, inspection, and testing:	
Reassembly Instructions:	
Packaging:	
Sterilization:	
Storage:	
Additional Information:	

Related Healthmark Products	
Other Product Support Documents	ProSys Instrument Care Product Brochure, ProSys Instrument Care Price List
Reference Documents	
Customer Service contact:	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2015-06-23 troom