

	VS-118-200	FIS-006SK	FIS-006SK	FIS-005-FIS-005SK	FIS-004	FIS-003	FIS-S001	FBR 001, FBR 002	MICR007
Description	Videoscope	Flexible Inspection Scope Kit	Cable	Flexible Inspection Scope Kit	2.4mm Video Borescope	Flexible Inspection Scope Kit	Flexible Inspection Scope Camera	Fiberscope and LED Light Source Adap.	USB Microscope
Glutaraldehyde	Y	Y		Y				Y	
Ortho-phthalaldehyde	Y	Y		Y				Y	
Peracetic Acid	Y	Y		Y					
Quaternary Ammonia Wipes	Y	Y		Y	Y	Y	Y	Y	Y
Enzymatic Detergent	Y	Y		Y				Y	
Neutral Ph Detergents	Y	Y	Y	Y	Y		Y	Y	Y
Hydrogen Peroxide Wipes	Y	Y	Y	Y	Y	Y	Y	Y	Y
Liquid Hydrogen Peroxide	Y	Y	Y	Y				Y	
Alcohol Wipes	Y	Y	Y	Y	Y	Y	Y	Y	Y
Bleach Wipes						Y	Y		
Sodium Borate				Y				Y	
Sodium Xylene Sulfonate				Y				Y	
Tetrapotassium Pyrophosphate				Y				Y	
Trisodium Nitrilotriacetate				Y				Y	
Ethylene oxide				Y				Y	
STERRAD® Systems (Standard Cycle)				Y				Y	
Steris System 1E	Y	Y		Y				Y	
STERIS V-PRO® Low Temperature Sterilization Systems (NonLumen Cycle)	Y	Y		Y				Y	
Additional Comments		Control Box- Isopropyl Alcohol, Can Not Be Submerged		Camera Cable- Isopropyl Alcohol, Can Not Be Submerged	Isopropyl Alcohol			LED Light Source Adapter- Enzymatic and Neutral pH solutions	Ethyl Alcohol