



Figure 4



Figure 5



Figure 6



Figure 7



Figure 8



Figure 9



Figure 10

Steps for Use of Product

1. Loosen the vice at the bottom of the Flex Arm. **Fig.1**
2. Secure the arm to table by adjusting the vice until the base is locked in position and stable. **Fig. 2**
3. Make sure the microphone holder is secured into place. The black button on the side of the neck should be clicked into place. **Fig. 3**
4. To remove the neck of the microphone holder, press the black button on the side of the neck that holds the microphone holder then lift the neck and the microphone holder up. **Fig. 4**
5. To reattach the neck, press the black button inward and slide the neck of the microphone holder down into the groove of the Flex Arm. Make sure the button clicks back into place (lift the neck up to ensure it is securely locked in).
6. To detach the neck of the microphone holder from the Flex Arm, grip the neck and loosen the silver bolt and neck counterclockwise, lift up from the Flex Arm. **Fig. 5**
7. Place the flexible working length through the microphone holder and place the handpiece inside of the holder. **Fig. 6**
8. Make sure the handpiece is secured inside the microphone holder with the buttons facing upwards. **Fig. 7**
9. Adjust the Flex Arm into the desired position. **Fig. 8**
Next, tilt the microphone holder up or down to the desired position for the FIS-005. **Fig. 9**
10. To remove the FIS-005 from the microphone holder, grip the back of the FIS-005 handpiece then pull backwards to release from the holder until the flexible working length is completely removed from the holder. **Fig. 10**

Interpretation of Test Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	<ul style="list-style-type: none"> • Use with Isopropyl Alcohol wipes per the wipe manufacturer’s IFU.
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	
Storage	
Additional Information	
Related Healthmark Products	
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2021-03-08 Suzanne Latta