

Instructions for Use: Flexible Arm for Fiberscope

Brand Name of Product	Flexible Arm for Fiberscope
Generic Name of Product	Flex arm
Product Code Number(s)	100657-FBR
Intended Use	To hold the Fiberscope in place while visually inspecting for residual soil inside cannulated items.
Range of Applications for Product	Healthmark's Fiberscope
Key Specifications of Product	<ul style="list-style-type: none"> • Flexible steel arm • Base fitting C-Clamp • Matte black vinyl • 2 feet long • Holder for Fiberscope

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Contents	
Shelf Life	



Instructions for Using Product	
Description of Use (s)	To hold the Fiberscope in place to visually inspect for remaining soils inside of cannulated items.
Preparation for Use	<ol style="list-style-type: none"> 1. Remove Flex Arm from packaging. 2. Straighten the Flex Arm so it stands vertically.
Diagrams (drawings, pictures)	<div style="display: flex; justify-content: space-around; align-items: center;"> <div style="text-align: center;">  <p>Figure 1</p> </div> <div style="text-align: center;">  <p>Figure 2</p> </div> </div>



Figure 3



Figure 4



Figure 5



Figure 6



Figure 7



Figure 8



Figure 9



Figure 10

Steps for Use of Product

1. Loosen the vice at the bottom of the Flex Arm. **Fig.1**
2. Secure the arm to table by adjusting the vice until the base is locked in position and stable. **Fig. 2**
3. Make sure the microphone holder is secured into place. The black button on the side of the neck should be clicked into place. **Fig. 3**
4. To remove the neck of the microphone holder, press the black button on the side of the neck that holds the microphone holder then lift the neck and the microphone holder up. **Fig. 4**
5. To reattach the neck, press the black button inward and slide the neck of the microphone holder down into the groove of the Flex Arm. Make sure the button clicks back into place (lift the neck up to ensure it is securely locked in).
6. To detach the neck of the microphone holder from the Flex Arm, grip the neck and loosen the silver bolt and neck counterclockwise, lift up from the Flex Arm. **Fig. 5**
7. Slide the Fiberscope into the microphone holder. **Fig. 6**
8. Make sure the handpiece is secured inside the microphone holder. **Fig. 7**
9. Adjust the Flex Arm into the desired position. **Fig. 8**
10. Next, tilt the microphone holder up or down to the desired position for the FBR - 001, FBR-002. **Fig. 9**

	11. To remove the Fiberscope from the microphone holder, grip the eye piece of the Fiberscope then pull backwards from the holder until it is completely detached from the holder. Fig. 10
Interpretation of Test Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	<ul style="list-style-type: none"> • Use with Isopropyl Alcohol wipes per the wipe manufacturer’s IFU.
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	
Storage	
Additional Information	
Related Healthmark Products	
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price list
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2019-07-24 Suzanne Latta

Rev. A