



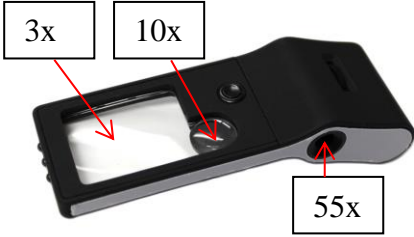

Instructions for Use: Multi-Magnifier

Brand Name of Product	Multi-Magnifier
Generic Name of Product	Multi-Magnification Magnifier
Product Code Number(s)	MAG-310
Purpose of Product	Small size that makes it convenient to carry at all times. Designed to allow for easy and thorough inspection of items. All in one magnifier magnifies by 3x, 10x and 55x
Range of Applications for Product	Items that need up close visual inspection for possible damage or residual soiling after reprocessing.
Key Specifications of Product	<ul style="list-style-type: none"> • 3x, 10x and 55x magnification • Lightweight and durable design • Built in lighting • Includes battery- (3) AG10/389/390 button batteries

Shipping & Storage

Shipping Conditions & Requirements	
Storage Conditions	
Packaging Contents	
Shelf Life	

Instructions for Using Product

Description of Use (s)	Close-up visual inspection of items after reprocessing
Preparation for Use	
Diagrams (drawings, pictures)	 
Steps for Use of Product	<ol style="list-style-type: none"> 1. After reprocessing; use pocket magnifier at optical inspection stations. 2. Select lens for use during optical inspection by amount needed. Magnifies by 3x, 10x and 55x. 3. Press button on side to turn on light source. Each time you press the button the light source will change light intensity and location of the lights that illuminate.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions

Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	<ul style="list-style-type: none"> • Lenses: clean lens cleaner and a soft cloth. • Rest of the magnifier: premoistened IPA wipe.
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	
Storage	
Additional Information	

Related Healthmark Products	
Other Product Support Documents	
Reference Documents	ProSys™ Brochure, ProSys™ Price List
Customer Service Contact	Healthmark Industries Company, Inc 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2020-08-19 Ashlynn Basile