

## Instructions for Use: LED Magnifier 4x

<b>Brand Name of Product</b>	LED Magnifier 4x
<b>Generic Name of Product</b>	LED Magnifier 4x
<b>Product Code Number(s)</b>	MAG-002
<b>Intended Use</b>	A tool to help with visual inspections with three levels of brightness and a 4x magnification for easy and thorough inspection of items.
<b>Range of Applications for Product</b>	Items, including robotic arms, that need up close visual inspection for possible damage or residual soiling after processing.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>● LED magnifier size- 85x85x60mm</li> <li>● 4x magnification</li> <li>● Lightweight</li> <li>● White 4 LED lights</li> <li>● Touch activated with 3 levels of brightness</li> <li>● On/Off switch</li> <li>● Microfiber lens cloth</li> <li>● Micro-USB Type C charging cable</li> <li>● Integrated battery</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Contents</b>	
<b>Shelf Life</b>	

<b>Instructions for Using Product</b>	
<b>Description of Use (s)</b>	A close-up visual inspection of items, including robotic arms, after reprocessing.
<b>Preparation for Use</b>	<p><b>Charge the magnifier before first use.</b></p> <p>Connect the charging cord with the lightning connector on the side of the magnifier and plug the USB end of the charging cord into a USB port or charger.</p> <p>The dot to the left of the connector will show red while charging and turns green when it is fully charged. The rechargeable battery has over 300 charges.</p>
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. Turn power switch on and use the touch activated setting (black circle) on top of the magnifier and select the level of brightness you would like to use.</li> <li>2. There are three levels of brightness:             <ul style="list-style-type: none"> <li>First touch- Low- low brightness- Max 8 hours of working time</li> <li>Second touch- Middle- middle brightness- Max 3 hours of working time</li> <li>Third touch- High- brightest- Max 1.5 hours of working time</li> </ul> </li> <li>3. Once you have selected the LED light setting, slide an item under the magnifier to visually inspect for cleanliness or damage through the glass of the magnifier.</li> </ol>
<b>Interpretation of Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	<ul style="list-style-type: none"> <li>● Turn power off when not in use.</li> <li>● Fire hazard. Keep magnifier away from direct sunlight to prevent accidental fire.</li> <li>● Do not stare directly at LED lights as this might damage your eyes.</li> <li>● Keep the magnifier away from heat sources or direct impact to prevent damage to the battery.</li> <li>● Do not manipulate the magnifier with wet hands.</li> <li>● Do not use if broken, may cause injuries.</li> </ul>
<b>Disposal</b>	

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	

<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	<ul style="list-style-type: none"> <li>● Clean lens with a soft clean microfiber cloth rubbing gently in a circular motion.</li> <li>● Moisten the lens cloth with water if necessary.</li> <li>● Do not use harsh chemicals, solvents or abrasives as they may damage the lens.</li> </ul>
<b>Cleaning – Automated</b>	
<b>Disinfection</b>	Wipe down the stand of the magnifier with pre-moistened alcohol wipe.
<b>Drying</b>	
<b>Maintenance, Inspection, and Testing</b>	
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	
<b>Storage</b>	
<b>Additional Information</b>	
<b>Related Healthmark Products</b>	
<b>Other Product Support Documents</b>	ProSys™ Brochure, ProSys™ Price List
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc 18600 MayIn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> hmark.com

2021-03-08 Suzanne Latta