








Instructions for Use: Light Cord Adapters

| | |
|--|--|
| Brand Name of Product | Light Cord Adapters |
| Generic Name of Product | Light Cord Adapters |
| Product Code Number(s) | LCA-001, LCA-002, LCA-003, LCA-004 |
| Intended Use | To test the light cord for intensity of the light and fiber optic damage. |
| Range of Applications for Product | Testing for intensity of light and damage. |
| Key Specifications of Product | <ul style="list-style-type: none"> • LCA-001- Storz Cord Adapter • LCA-002- Stryker Cord Adapter • LCA-003- ACMI Cord Adapter • LCA-004- Wolf Cord Adapter |

| Shipping & Storage | |
|---|-----|
| Shipping Conditions & Requirements | N/A |
| Storage Conditions | N/A |
| Packaging Contents | N/A |
| Shelf Life | N/A |

| Instructions for Using Product | |
|--------------------------------------|---|
| Description of Use(s) | To test the light cord for intensity of the light and fiber optic damage. |
| Preparation for Use | N/A |
| Diagrams (drawings, pictures) |    <p style="text-align: center;">Figure 1</p> <p style="text-align: center;">Figure 1a</p> |

| | | |
|--|--|---|
| |  |  |
| | Figure 2 | Figure 2a |
| |  |  |
| | Figure 3 | Figure 4 |
| Steps for Use of Product | <ol style="list-style-type: none"> 1. Choose the appropriate light cord adapter, attach to the light cord and secure together. Fig. 1, 1a 2. Attach the light source to the light cord adapter then secure together. Fig. 2, 2a 3. Click the button on top of the light source and choose between a high, medium, or low light and check for intensity of the light at the end of the light cord as well as inspecting for defects. Fig. 3 4. Next, check for fiber optic damage in the light cord. Fig. 4 5. Any defects or damage remove the light cord from service. | |
| Interpretation of Test Results | N/A | |
| Contraindications of Test Results | N/A | |
| Documentation | N/A | |
| Special Warnings and Cautions | N/A | |
| Disposal | N/A | |

| | |
|---|---|
| Reprocessing Instructions | |
| Point of Use | N/A |
| Preparation for Decontamination | N/A |
| Disassembly Instructions | N/A |
| Cleaning – Manual | N/A |
| Cleaning – Automated | N/A |
| Disinfection | Wipe down adapter with an alcohol wipe as needed. |
| Drying | N/A |
| Maintenance, Inspection, and Testing | N/A |
| Reassembly Instructions | N/A |
| Packaging | N/A |
| Sterilization | N/A |
| Storage | N/A |
| Additional Information | |
| Related Healthmark Products | EndoPro-Cam, Light Source |
| Other Product Support Documents | ProSys™ Brochure, ProSys™ Price List |
| Reference Documents | N/A |
| Customer Service Contact | Healthmark Industries Company, Inc 18600 Malyn Blvd. Fraser, MI 48026 |

| | |
|--|---|
| | 1-586-774-7600 healthmark@hmark.com hmark.com |
|--|---|

2021-04-08 Suzanne Latta