

Instructions for Use: Loupe Set with Smart Phone Adapter

Brand Name of Product	Loupe Set with Smart Phone Adapter
Generic Name of Product	Loupe Set with Smart Phone Adapter
Product Code Number(s)	LUP-004
Intended Use	To enhance visual inspection with magnification for hard to see areas in lumened/channeled instruments.
Range of Applications for Product	Use with a smart phone for magnified visual inspection.
Key Specifications of Product	<ol style="list-style-type: none"> 1 Smartphone Clip 4 eye loupes with different magnification 4.5x, 6.5x, 8x, and 13x eye loupes Ergonomic design

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Contents	
Shelf Life	

Instructions for Using Product	
Description of Use (s)	To enhance visual inspection with magnification for lumened instruments.
Preparation for Use	<ol style="list-style-type: none"> 1. Pull off and remove the rubber eyecup from loupe. 2. Slide phone clip onto the lip of the loupe and if needed rotate clockwise until it snaps into place. 3. Attach phone clip to top of phone with loupe centered over the rear camera lens. (For dual cameras use the primary one. Apple dual camera phones, the primary camera is the one closest to the corner).
Diagrams (drawings, pictures)	
Steps for Use of Product	<p>Using Loupe with Phone Clip</p> <ol style="list-style-type: none"> 1. Open the camera app in either video (recommended) or photo modes. 2. Adjust loupe position until image is centered on screen. 3. Once centered you may pinch and zoom image so you no longer see a black edge around camera view. 4. Adjust the distance between the loupe and viewing subject until focused. 5. Adjust the distance between the loupe and viewing subject until focused. <p>Removing Phone Clip</p> <ol style="list-style-type: none"> 1. Remove clip from phone. 2. Remove loupe from clip by rotating counter-clockwise while holding the phone clip. 3. Replace rubber eyecup onto loupe by hooking one edge of eyecup onto the lip of the loupe and stretching eyecup over the other half of the lip. 4. Store eye loupe set in gift when not in use.
Interpretation of Test Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	<p>Do not use abrasives, chemical cleaners or solvents.</p> <p>To avoid possible permanent eye damage, do not view the sun or any bright light source with the loupes.</p>
Disposal	

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	Clean with a microfiber cloth.

Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	
Storage	
Additional Information	
Related Healthmark Products	
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2021-03-08 Suzanne Latta