




Brand Name of Product	Made in USA LED Magnifier
Generic Name of Product	Magnifying Lamp
Product Code Number(s)	42400-4 RD, 82400-4BL, 11426
Intended Use	Designed for professional optical inspection of items and equipment.
Range of Applications for Product	Any products that require optical inspection after washing and before being sterilized.
Key Specifications of Product	<ul style="list-style-type: none"> • 7" X 5¼" RECTANGULAR LED MAGNIFIER - 43" REACH - TABLE EDGE CLAMP (Blue) • 6" ROUND LED MAGNIFIER - 43" REACH -SCREW DOWN BASE – Red • Assembled with low voltage (15-24v) power supplies. Shuts off every 12 hours to curb power supplies. • 3" wide with corner holes mount (Sold Separately) • 2x magnification

Shipping & Storage

Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	
Shelf Life	

Instructions for Using Product

Description of Use (s)	Designed for visual inspection for items and equipment.
Preparation for Use	
Diagrams (drawings, pictures)	
Steps for Use of Product	<ol style="list-style-type: none"> 1) Attach the lamp to the desired working surface. If using the optional bracket to mount please note: <ol style="list-style-type: none"> a. Mounting to typical wall with 2"x 4" studs, there will be a thin side of the stud, approximately 1.75" to anchor to. The holes will not allow for all four to be anchored on a stud this size. It is also NOT advisable to anchor this directly to dry wall as it would probably fall off. 2) Position the lamp in the desired position and turn on using the on-off switch.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions

Point of Use	
---------------------	--

Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	<ul style="list-style-type: none"> • Lenses are glass and much more scratch resistant than plastic lenses. To avoid scratching highly polished lens surfaces: <ul style="list-style-type: none"> ○ Clean with a soft cloth moistened in warm, soapy water or use a lens cleaner. ○ Dry with clean soft cloth (never use abrasive cleaners).
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	
Storage	
Additional Information	

Related Healthmark Products	
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2018-06-15 msmith