



Brand Name of Product	OptiVisor
Generic Name of Product	Headband magnified visor
Product Code Number(s)	26106
Purpose of Product	This product is a precision headband magnifying visor glass lens that improves vision and increases your efficiency. It's used for optical inspection when magnification is needed to visually inspect products to ensure they are properly cleaned.
Range of Applications for Product	Any products that need visual inspection after cleaning.
Key specifications of product	<ul style="list-style-type: none"> • Adjustable headband • Selection of magnified lens available • Lightweight construction

Shipping & Storage

Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	
Shelf Life	

Instructions for Using Product

Description of Use (s)	Wear and use during visual inspection of surgical instruments
Preparation for Use	All of the lens plates are interchangeable. In order to change the lens plate push the fasteners out from the back, inside visor and snap the replacement lens plate.
Diagrams (drawings, pictures):	
Steps for Use of Product	<ol style="list-style-type: none"> 1. Position the headband by squarely placing on your head with the center of the lens plate at the bridge of the nose. 2. By adjusting the knob at the rear of the unit enough tension can be applied to hold the headband in place. 3. The adjustable pivots on either side of the visor should be tightened with enough tension to hold the lens in front of the eyes as well as up out of the path of vision when not in use.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	<ol style="list-style-type: none"> 1. The visor should never be raised, lowered, carried, or positioned by means of the lens. Doing so may cause damage both to the lens and the visor. 2. Never put pressure on the lens or area around the lens to position the visor as this may cause the lens to dislocate. 3. This product is not a safety device and should be worn over safety glasses when safety is a concern. The OptiVisor is designed to be worn over prescription eye glasses and not replace them.
Disposal	

Reprocessing Instructions

Point of use:	
Preparation for decontamination:	
Disassembly Instructions:	
Cleaning – Manual:	<ul style="list-style-type: none"> • The OptiVisor Lenses are glass and much more scratch resistant than plastic lenses. To avoid scratching highly polished lens surfaces: <ul style="list-style-type: none"> ○ Wash in warm soapy water or lens cleaner. ○ Dry with clean soft cloth (never use abrasive cleaners).
Cleaning – Automated:	
Disinfection:	
Drying:	
Maintenance, inspection, and testing:	
Reassembly Instructions:	

Packaging:	
Sterilization:	
Storage:	
Additional Information:	

Related Healthmark Products	
Other Product Support Documents	
Reference Documents	Prosys Brochure, Prosys Price List
Customer Service contact:	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2013-01-03 msmith