


## Instructions for Use: Non-Linting Disposable Wipes

<b>Brand Name of Product</b>	Non-linting Disposable Wipes
<b>Generic Name of Product</b>	Instrument Cleaning Wipes
<b>Product Code Number(s)</b>	NLW001, NLW002, NLW003
<b>Purpose of Product</b>	Non-linting wipes are for use immediately after the item has been used, and prior to automated or manual disinfection to ensure items are dry.
<b>Range of Applications for Product</b>	Any item that requires immediate removal of soils after procedure.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• Non-linting blended poly-cellulose material</li> <li>• Single-use</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Contents</b>	
<b>Packaging Contents</b>	
<b>Shelf Life</b>	

<b>Instructions for Using Product</b>	
<b>Description of Use(s)</b>	To allow for safe cleaning of sensitive items and fixtures, as well as all general surfaces.
<b>Preparation for Use</b>	<ul style="list-style-type: none"> <li>• Follow the item manufacturer's IFU.</li> <li>• If utilizing a cleaning agent, be sure to follow the detergent manufacturer's IFU for proper preparation of the cleaning solution.</li> </ul>
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. Moisten with cleaning solution (if applicable).</li> <li>2. Wipe down instrument to remove soils and gross debris.</li> <li>3. Dispose after use on a single instrument.</li> </ol>
<b>Interpretation of Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	<ul style="list-style-type: none"> <li>• Consult the IFU of the item manufacturer to ensure compliance with their instructions.</li> <li>• Pay attention to area(s) of the item that must be free from moisture.</li> <li>• Pay attention to chemical compatibility of the item and areas of the item with the cleaning agent you select.</li> </ul>
<b>Disposal</b>	Treat as biohazard and discard of in compliance with facility guidelines.

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	
<b>Cleaning – Automated</b>	
<b>Disinfection</b>	
<b>Drying</b>	
<b>Maintenance, Inspection and Testing</b>	
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	
<b>Storage</b>	
<b>Additional Information</b>	

<b>Related Healthmark Products</b>	DuraSponge™, Sterile Wipes
<b>Other Product Support Documents</b>	ProSys™ Brochure, ProSys™ Price List
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>

2021-02-18 msmith